Stat Profile® Prime VET CCS Comp Calibrator Cartridge

Kalibratorkassette, Καρέτα Βαθμονομή, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett, Kalibratorkazetta, 교정기 카트리지, キャリブレータカートリッジ, 校准液盒

LOT 23139021

2024-08-02

CAL A	≥ 500 mL	CAL B	≥ 250 mL	CAL F	≥ 720 mL	SOLN R	≥ 300 mL
pH	7.355	pH	6.840	PCO ₂	50.7 mmHg	KCl	2 mol/L
PCO ₂	24.7 mmHg	PO ₂	167.0 mmHg	PO ₂	92.7 mmHg		
Na ⁺	133.0 mmol/L	Na ⁺	68.5 mmol/L				
K ⁺	4.0 mmol/L	K ⁺	10.0 mmol/L				
Cl ⁻	106.0 mmol/L	Cl ⁻	46.0 mmol/L				
iCa	1.10 mmol/L	iCa	2.20 mmol/L				
Glu	80 mg/dL	Glu	200 mg/dL				
Lac	2.0 mmol/L	Lac	10.0 mmol/L				

(EN)

Product Description:

The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions with individual flexible bags housed in a cardboard box: 3 Calibrators (Calibrator A, B, and F), reference solution, and a flexible waste bag. Each bag includes a filament with a stopper. The exposed bag filament is arranged in a loop along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Indicated Use:

For the quantitative determination of Na, K, Cl, iCa, Glucose, Lactate, pH, PCO₂, PO₂, and Hct in blood using the Stat Profile Prime VET CCS Comp Analyzer.

Methodology:

Refer to Prime Vet CCS Comp Analyzer Instructions for Use Manual for Test Methodologies and Principles.

Compliance:

Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

For Veterinary Use Only. Not for Human Use. Refer to the Prime Vet CCS Comp Analyzer Instructions for Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals, troubleshooting information, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, respiratory irritant).

Storage:

Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

Directions For Use:

Refer to Prime Vet CCS Comp Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards:

Analytes traced to NIST Standard Reference Materials.

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Waltham, MA 02454 U.S.A.