Nova 16

CONTROL 1 | 2

LOT 23198052

2024-12-12

Expected Ranges, Erwartete Bereiche, Αναμενόμενα Εύρη, Rangos esperados, Fourchettes attendues, Intervalli previsti, Gamas previstas, Förväntade områden

LOT			
CONTROL 1	23163059	2024-12-12	CONTROL 1
CONTROL 2	23163060	2024-12-12	min - \bar{x} - max
Na ⁺	mmol/L	141 - 146 - 151	116 - 119 - 123
K ⁺	mmol/L	3.4 - 3.6 - 3.8	5.7 - 6.0 - 6.3
Cl ⁻	mmol/L	97 - 101 - 105	72 - 76 - 80
TCO ₂	mmol/L	22 - 26 - 30	11 - 14 - 17
Glu	mg/dL	73 - 80 - 87	253 - 277 - 301
Glu	mmol/L	4.02 - 4.40 - 4.79	13.92 - 15.24 - 16.56
BUN	mg/dL	11 - 15 - 19	43 - 50 - 57
BUN	mmol/L	8 - 11 - 14	31 - 36 - 41
Urea	mg/dL	24 - 32 - 41	92 - 107 - 122
Urea	mmol/L	4 - 5 - 7	15 - 18 - 20
Li ⁺	mmol/L	0.49 - 0.61 - 0.73	1.75 - 2.00 - 2.25
Creat	mg/dL	0.8 - 1.1 - 1.4	5.8 - 6.5 - 7.2
Creat	μmol/L	71 - 97 - 124	513 - 575 - 637

Product Description

Contains assayed quality control material formulated at three levels for monitoring the measurement of Na⁺, K⁺, Cl⁻, Li⁺, TCO₂, Glucose, BUN (urea) and creatinine.

Intended Use

For in vitro diagnostic use to monitor the performance of the Nova 16 Analyzers.

Methodology

Refer to Nova 16 Analyzer Instruction for Use Manual for Methodology and Principles.

Composition

Nova Control is a pale straw colored liquid assayed quality control material. Contains organic and inorganic constituents in a bovine albumin base. A preservative has been added to the product; however it does not contain stabilizers, viscosity adjusters, drug analogs or other materials which might affect the performance of the sensor. Nova Control contains a buffered solution with known concentrations of Na⁺, K⁺, Cl⁻, Li⁺, TCO₂, Glucose, Urea and creatinine.

Each ampule contains a minimum volume of 1.2 mL.

Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials.

Warning and Precautions

For in vitro diagnostic use. Store at or below -15°C. Do not store at room temperature. Refer to the Nova Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents.

Storage

Store at or below -15°C. Each ampule has a lot number and expiration date printed on the label.

Directions for Use

Re-constitute the freeze solution by thawing to room temperature. Re-suspend the ampule contents. Nova Controls contain proteins, which may settle with storage. Gently rotate each ampule b/w between fingers or hands to mix contents prior to use. Avoid shaking, excess foaming may occur.

Ship Nova Controls in a padded envelope with a gauze or gloves, and within ONE MINUTE, aspirate solution with sample probe or transfer solution into sample cups with end-evaporation caps. Verify that the lot number appearing on the expected ranges table is the same as the lot number printed on the material.

Limitations

Assigned assay values will be adversely affected by improper handling or storage, or if the integrity of the assay is compromised prior to use.

To prevent contamination and loss of TCO₂, open ampules over a waste solution. If using the auto sampler, the use of anti-evaporation caps is recommended to prevent TCO₂ loss. Cloudiness or particulate matter in the solution is an indication of possible deterioration.

For use Nova Biomedical. Only performance characteristics have not been established for use on other manufacturer's analyzers.

This product is warranted to perform as described in its product insert, and Nova Biomedical disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall Nova Biomedical be liable for any consequential damages arising out of the aforesaid express warranty.

Traceability of Standards

Analyses are traced to NIST Standard reference materials.

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