
REF 34085, 34086, 34087, 34088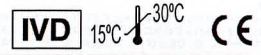

Stat Profile® pHox® Plus L Calibrator Cartridge

Kalibratorkassette, Κασέτα Βαθμονομητή, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett

CAL A ≥ 300 mL		CAL B ≥ 250 mL		CAL C ≥ 100 mL		CAL D ≥ 100 mL		SOLN R ≥ 300 mL	
Na ⁺	143 mmol/L	HCO ₃ ⁻	46.84 mmol/L	pH	7.335 @ 37.0°C	pH	6.791 @ 37.0°C	KCl	2 mol/L
HCO ₃ ⁻	24.92 mmol/L			Ca ⁺⁺	2.04 mmol/L	Glu	1000 mg/dL		
K ⁺	4.06 mmol/L			Na ⁺	73 mmol/L	Lac	12 mmol/L		
Cl ⁻	98 mmol/L			K ⁺	10.12 mmol/L				
Ca ⁺⁺	1.02 mmol/L			Cl ⁻	53 mmol/L				
Glu	40 mg/dL			Glu	200 mg/dL				
Lac	1.0 mmol/L			Lac	6.0 mmol/L				

Product Description
The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: four Calibrators (Calibrator A, B, C, and D), reference solution, and a flexible waste bag. Each bag includes a filament with a septa. The exposed bag filaments are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use
For the quantitative determination of Na⁺, K⁺, Cl⁻/Ca⁺⁺, Glucose, Lactate, pH, P_{CO₂}, PO₂, SO₂, Hb, and Hct in human blood using the Nova Biomedical Stat Profile pHox Plus L Analyzer.

Methodology
Refer to Stat Profile pHox Plus L Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition
Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2)

Warnings and Cautions:
Intended for in vitro diagnostic use.
Refer to the Stat Profile pHox Plus L Analyzer Instructions for Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals.
Troubleshooting information, Methodology and Principles of the testing procedures.
Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guidelines.
Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

Directions For Use
Refer to the Stat Profile pHox Plus L Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards
Analysis traced to NIST Standard Reference Materials.

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