Stat Profile Prime Plus® Calibrator Cartridge

Stat Profile Prime Plus® Kalibrierkassette, Φυσικό μέσου βαθμονόμησης με κρυστάλλινη Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus®, Cartuccia calibratore Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Stat Profile Prime Plus® kalibrátor patron, Stat Profile Prime Plus® קרטיזן Stat Profile Prime Plus®, Stat Profile Prime Plus® カリブレーターカートリッジ, Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 校准试剂盒

LOT 23283016

2025-03-22

CAL A	CAL B	CAL C	CAL D	CAL F	SOLN R
HCO ₃	11 mmol/L	pH	6.834 @ 37°C	HCO ₃	22 mmol/L
Na ⁺	135.5 mmol/L	H ⁺	146.55 nmol/L	Cl ⁻	69.0 mmol/L
K ⁺	4.0 mmol/L	Na ⁺	77.0 mmol/L	Glu	200 mg/dL
Cl ⁻	108.0 mmol/L	K ⁺	10.0 mmol/L	Glu	11.10 mmol/L
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L
Glu	80 mg/dL	tHb	15.5 g/dL		
Glu	4.44 mmol/L	tHb	155 g/L		
Lac	2.0 mmol/L	tHb	9.6 mmol/L		
Lac	17.8 mg/dL				
BUN	10 mg/dL				
BUN	3.6 mmol/L				

(EN)

Description

The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F SOLN), Reference Solution, and a flexible waste bag. Each bag has a septum with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use

For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lacatate, pH, PCO₂, PO₂ %, Hct/Hb, HbF⁻, O₂Hb, COHb, MetHb, HHb and tBil⁺ in human blood using the Stat Profile Prime Plus Analyzer.

*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

Methodology

Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition

The calibration are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dyes. Contain no constituents of human origin, however, good laboratory practices should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions

Intended for *in vitro* diagnostic use.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents.

Traceability of Standards

Analytes traced to NIST Standard Reference Materials wherever possible.

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EC REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.