Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrerkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιγό μέσου βαθμούμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininné, Stat Profile Prime Plus® カリブレーターカートリッジ (クリエチニン), 크레이티닌 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 23290102

2025-04-06

CAL A	CAL B	CAL C	CAL D	CAL F	SOLN R
HCO ₃ 11 mmol/L	pH 6.834 @ 37°C	HCO ₃ 22 mmol/L	pH 7.354 @ 37°C	PO ₂ 104.2 mmHg	KCl 2 mol/L
Na ⁺ 135.5 mmol/L	H ⁺ 146.55 nmol/L	Cl ⁻ 69.0 mmol/L	H ⁺ 44.26 nmol/L	PO ₂ 13.86 kPa	 WASTE CONTAINER W
K ⁺ 4.0 mmol/L	Na ⁺ 77.0 mmol/L	Glu 200 mg/dL	Glu 11.10 mmol/L		
Cl ⁻ 108.0 mmol/L	K ⁺ 10.0 mmol/L	Lac 10.0 mmol/L	Lac 89.0 mg/dL		
iCa 1.04 mmol/L	iCa 1.92 mmol/L	BUN 50 mg/dL	BUN 17.9 mmol/L		
iCa 4.16 mg/dL	iCa 7.68 mg/dL	Creat 5.0 mg/dL	Creat 0.44 mmol/L		
iMg 0.50 mmol/L	iMg 1.50 mmol/L	tHb 15.4 g/dL	Creat 442 µmol/L		
iMg 1.22 mg/dL	iMg 3.63 mg/dL				
Glu 80 mg/dL	tHb 154 g/L				
Glu 4.44 mmol/L					
Lac 2.0 mmol/L					
Lac 17.8 mg/dL					
BUN 10 mg/dL					
BUN 3.6 mmol/L					
Creat 1.0 mg/dL					
Creat 0.09 mmol/L					
Creat 88.4 µmol/L					

(EN)

Description
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference Solution, and a flexible waste bag. Each bag includes a fitment with a septum. The enclosed bags must be inserted in a line after the rear of the cartridge body. The septa during the insertion of the bags must be inserted into the analyzers. A second set of syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use

For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO₂, PO₂, SO₄²⁻, Hct, tHb, HbF⁺, O₂Hb, COHb, MetHb, HHb, and tBil⁺ in whole blood using the Stat Profile Prime Plus Analyzer.

Methodology

Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition

The Calibrators are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dye. Contain no constituents of human origin, however, good laboratory practice should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions

Intended for in vitro diagnostic use.
Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use**Activation of Cartridge**

Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the fitments on the carton for Cal A and Cal C.

1. Remove the cap with tip side down and remove protective cap.

2. Attach one of the enclosed needle assemblies to the syringe. Remove the protective cover from the needle.

3. Match the color and label of the syringe to the appropriate fitment and insert needle.

4. Gently push the plunger until the contents are dispensed.

5. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.

6. Remove needle/syringe assembly from fitment and discard in an appropriate sharps container.

7. Mix cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions For Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

*Not available in the USA or Point-of-Care/Near-Patient Testing use.

Traceability of Standards

Analyses traced to NIST Standard Reference Materials wherever possible.

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EC REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.