Linearity Standard Set A

Linearitätsstandard Satz A, Πρότυπο Γραμμικότητας Set A, Estándar de linealidad Set A,
Set A standard de linéarité, Standard di linearità Set A, Conjunto A de padrão de linearidade, Linearity Standard Set A

LEVEL 6

Expected Ranges, Erwartete Bereiche, Αναμενόμενα Εύρη, Rangos esperados, Fourchettes attendues, Intervalli previsti, Gamas previstas, Förväntade områden			
LOT	23298037	2025-04-25	LEVEL 6
NOVA 1-16	Na ⁺ K ⁺ Cl ⁻	mmol/L mmol/L mmol/L	90 - 110 90 - 110 180 - 220

Product Description
 Aquous solution containing Na⁺, K⁺, and Cl⁻. For use on NOVA analyzers ONLY.

Intended Use
 For in vitro diagnostic use with NOVA Analyzers to verify calibration, analytical linearity, estimate test imprecision, and detect systematic analytical deviations that may arise from calibrator or analytical instrument variation.

Methodology
 Refer to Nova Analyzer Instructions for Use for Methodology and Principles of the testing procedures.

Composition
 Buffer solutions containing Na⁺, K⁺ and Cl⁻. Minimum volume 50 ml.
 Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions
 Incomplete dissolution of reagents.
 Cloudiness or particulate matter in the solution is an indication of possible deterioration. DO NOT FREEZE.
 Follow standard practices for handling laboratory reagents. Discard in accordance with local requirements.

Storage
 Store at 15-30°C. After opening, the solution is stable for 6 months.

Directions for Use
 Linearitätsstandard are recommended for use as frequently as required by local regulatory and hospital requirements.

It is recommended that each laboratory establish the upper and lower limits of its reportable range for each sample type and devise a policy for processing specimens that fall outside of this range.

Limitations
 For use on Nova Biomedical analyzers only. Performance characteristics not established for use on other manufacturers analyzers.

Traceability of Standards
 Analyses traced to NIST Standard Reference Materials.

Expected Ranges
 The expected range indicates the maximum deviations from the mean value that may be expected under different analytical conditions for instruments operating within specifications. The expected ranges for the analytes listed in the table above were verified using replicate determinations on Nova analyzers.

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