REF 58404, 58405

15°C 30°C CE

Stat Profile Prime Plus® VET Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® VET Kalibrierkassette mit Kreatinin, Φυσιγύο μέσου βαθμονόμησης VET με κρεατινίνη Stat Profile Prime Plus®, Cartucho de calibrador con creatinina Stat Profile Prime Plus® VET, Cartouche de calibrateurs avec créatinine Stat Profile Prime Plus® VET, Cartuccia di calibratore con creatinina Stat Profile Prime Plus® VET com creatinina, Stat Profile Prime Plus® VET kalibrátor patron kreatininnel, Stat Profile Prime Plus® VET カリブレーター(クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® VET 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 23313074

2025-04-19

CAL A	≥ 495 mL	CAL B	≥ 250 mL	CAL C	≥ 245 mL	CAL D	≥ 250 mL	CAL F	≥ 850 mL	SOLN R	≥ 500 mL
HCO ₃	11 mmol/L	pH	6.834 @ 37°C	HCO ₃	22 mmol/L	pH	7.354 @ 37°C	PO ₂	106.7 mmHg	KCl	2 mol/L
Na ⁺	135.5 mmol/L	H ⁺	146.55 nmol/L	Cl ⁻	69.0 mmol/L	H ⁺	44.26 nmol/L	PO ₂	14.19 kPa		
K ⁺	4.0 mmol/L	Na ⁺	77.0 mmol/L	Glu	200 mg/dL					!	WASTE CONTAINER W
Cl ⁻	108.0 mmol/L	K ⁺	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.4 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	154 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.6 mmol/L	Creat	442 µmol/L						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 µmol/L										

EN

Description
The kit contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use
For the quantitative determination of Na⁺, K⁺, Cl⁻, Ca²⁺, Mg²⁺, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO₂, PO₂, SO₄²⁻, Hct, tHb, O₂Hb, COHb, MetHb, HHb, and tBil in animal whole blood using the Stat Profile Prime Plus VET Analyzer.

Methodology
Refer to Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for Methodologies and Principles.

Compatibility
Calibrator and solution volumes are listed in the table. Contains no constituents of human origin; however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines. Vaste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use
Activation of Cartridge
Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the cartridge for identification.

- Hold the syringe with tip side down and remove protective cap.
- Attach one of the enclosed needle assemblies to the syringe. Remove the protective cover from the needle.
- Match the color and label of the syringe to the appropriate filament and insert needle.
- DO NOT PUSH BACK ON THE PLUNGER TO FLUID CONTENTS OF SYRINGE.
- Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
- Repeat steps 1-5 for the next Cal.
- Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards
Analytes traced to NIST Standard Reference Materials wherever possible.

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Waltham, MA 02454 U.S.A.