Stat Profile® Prime VET CCS Comp Calibrator Cartridge

Kalibratorkassette, Καρέτα Βαθμονομής, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett, Kalibratorkazetta, 교정기 카트리지, キャリブレータカートリッジ, 校准液盒

LOT 23355036

2025-05-30

CAL A	$\geq 500 \text{ mL}$	CAL B	$\geq 250 \text{ mL}$	CAL F	$\geq 720 \text{ mL}$	SOLN R	$\geq 300 \text{ mL}$
pH	7.352	pH	6.840	PCO_2	52.2 mmHg	KCl	2 mol/L
PCO_2	24.9 mmHg	PO_2	167.0 mmHg	PO_2	97.2 mmHg		
Na^+	133.0 mmol/L	Na^+	68.5 mmol/L				
K^+	4.0 mmol/L	K^+	10.0 mmol/L				
Cl^-	106.0 mmol/L	Cl^-	46.0 mmol/L				
iCa	1.10 mmol/L	iCa	2.20 mmol/L				
Glu	80 mg/dL	Glu	200 mg/dL				
Lac	2.0 mmol/L	Lac	10.0 mmol/L				

WASTE CONTAINER W

(EN)

Product Description:

The above table lists the concentrations of constituents for the calibrator and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: 3 Calibrators (Calibrator A, B, and F), reference solution, and a flexible waste bag. Each bag includes a fitting with a septa. The exposed bag fittings are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use:

For the quantitative determination of Na, K, Cl, iCa, Glucose, Lactate, pH, PCO_2 , PO_2 , and Hct in blood using the Stat Profile Prime VET CCS Comp Analyzer.

Methodology:

Refer to Prime Vet CCS Comp Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition:

Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

For Veterinary Use Only, Not for Human Use. Refer to the Prime Vet CCS Comp Analyzer Instructions For Use Manual for complete directions for use. Installation instructions, recommendations for calibration intervals, Troubleshooting information, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage:

Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

Directions For Use:

Refer to the Prime Vet CCS Comp Analyzer Instructions For Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals, Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards:

Analytes traced to NIST Standard Reference Materials.

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