Stat Profile Prime Plus® Calibrator Cartridge

Stat Profile Prime Plus® Kalibrierkassette, Φυσιγγιό μέσου βαθμονόμησης με κραστινήν Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus®, Cartuccia calibratore Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Stat Profile Prime Plus® kalibrátor patron, 712 711 712 713 Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレーターカートリッジ, Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 校准试剂盒

LOT 24018036

2025-06-27

| CAL A | CAL B | CAL C | CAL D | CAL F | SOLN R |
|---------------------------------|---------------------------------|--------------------------------|--------------------------------|-------------------------------|----------------|
| HCO ₃ 11 mmol/L | pH 6.834 @ 37°C | HCO ₃ 22 mmol/L | pH 7.354 @ 37°C | PO ₂ 104.5 mmHg | KCl 2 mol/L |
| Na ⁺ 135.5 mmol/L | H ⁺ 146.55 nmol/L | Cl ⁻ 69.0 mmol/L | H ⁺ 44.26 nmol/L | PO ₂ 13.90 kPa | |
| K ⁺ 4.0 mmol/L | Na ⁺ 77.0 mmol/L | Glu 200 mg/dL | | | |
| Cl ⁻ 108.0 mmol/L | K ⁺ 10.0 mmol/L | Glu 11.10 mmol/L | | | |
| iCa 1.04 mmol/L | iCa 1.92 mmol/L | Lac 10.0 mmol/L | | | |
| iCa 4.16 mg/dL | iCa 7.68 mg/dL | Lac 89.0 mg/dL | | | |
| iMg 0.50 mmol/L | iMg 1.50 mmol/L | BUN 50 mg/dL | | | |
| iMg 1.22 mg/dL | iMg 3.63 mg/dL | BUN 17.9 mmol/L | | | |
| Glu 80 mg/dL | tHb 15.5 g/dL | | | | |
| Glu 4.44 mmol/L | tHb 155 g/L | | | | |
| Lac 2.0 mmol/L | tHb 9.6 mmol/L | | | | |
| Lac 17.8 mg/dL | | | | | |
| BUN 10 mg/dL | | | | | |
| BUN 3.6 mmol/L | | | | | |

(EN)

Description
The Cartridge contains the following aqueous solutions within individual bags: 6 Calibrators (Calibrator A, B, C, D, and F SOLN), Reference Solution, and a flexible waste bag. Each bag includes a septum. The exposed bag fittings are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use
For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, pH, PCO₂, PO₂, SO₄%, Hct, tHb, HbF, O₂Hb, COHb, MetHb, HHb, and tBil¹ in human blood using the Stat Profile Prime Plus Analyzer.

¹Not available in the USA or for Point-of-Care/Near-Patient Testing use.

Methodology
Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

Compatibility
The calibrations are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dye. Contain no constituents of human origin, however, good laboratory practice should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Intended for *in vitro* diagnostic use.
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15–30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents.

Traceability of Standards
Analytes traced to NIST Standard Reference Materials wherever possible.

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