Stat Profile® Prime VET CCS Comp Calibrator Cartridge

Kalibratorkassette, Καρότα Βαθμονούμη, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkasset, Kalibratorkazetta, 교정 카트리지, キャリブレータカートリッジ, 校准液盒

LOT 24025042

2025-07-04

CAL A	$\geq 500 \text{ mL}$	CAL B	$\geq 250 \text{ mL}$	CAL F	$\geq 720 \text{ mL}$	SOLN R	$\geq 300 \text{ mL}$
pH	7.356	pH	6.840	PCO_2	50.4 mmHg	KCl	2 mol/L
PCO_2	24.8 mmHg	PO_2	167.0 mmHg	PO_2	96.5 mmHg		
Na^+	133.0 mmol/L	Na^+	68.5 mmol/L				
K^+	4.0 mmol/L	K^+	10.0 mmol/L				
Cl^-	106.0 mmol/L	Cl^-	46.0 mmol/L				
iCa	1.10 mmol/L	iCa	2.20 mmol/L				
Glu	80 mg/dL	Glu	200 mg/dL				
Lac	2.0 mmol/L	Lac	10.0 mmol/L				

(EN)

Product Description

The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: 3 Calibrators (Calibrator A, B, and F), reference bag, and a flexible waste bag. Each bag includes a luerfit with a septa. The exposed bag fittings are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge in the analyser.

Intended Use

For the quantitative determination of Na, K, Cl, iCa, Glucose, Lactate, pH, PCO_2 , PO_2 , and Hct in blood using the Stat Profile Prime VET CCS Comp Analyzer.

Methodology

Refer to the Prime Vet CCS Comp Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition

Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

For Veterinary Use Only; Not for Human Use. Refer to the Prime Vet CCS Comp Analyzer Instructions for Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals, Troubleshooting information, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

Directions For Use

Refer to the Prime Vet CCS Comp Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridges must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards

Analyses traced to NIST Standard Reference Materials.

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Waltham, MA 02454 U.S.A.