Stat Profile® Prime VET CCS Comp Calibrator Cartridge

Kalibratorkassette, Καρότα Βαθμονομή, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett, Kalibratorkazetta, 교정기 카트리지, キャリブレータカートリッジ, 校准液盒

LOT 24066045

2025-08-27

CAL A ≥ 500 mL	CAL B ≥ 250 mL	CAL F ≥ 720 mL	SOLN R ≥ 300 mL
pH 7.356	pH 6.840	PCO ₂ 51.0 mmHg	KCl 2 mol/L
PCO ₂ 24.5 mmHg	PO ₂ 167.0 mmHg	PO ₂ 98.0 mmHg	 WASTE CONTAINER W
Na ⁺ 133.0 mmol/L	Na ⁺ 68.5 mmol/L		
K ⁺ 4.0 mmol/L	K ⁺ 10.0 mmol/L		
Cl ⁻ 106.0 mmol/L	Cl ⁻ 46.0 mmol/L		
iCa 1.10 mmol/L	iCa 2.20 mmol/L		
Glu 80 mg/dL	Glu 200 mg/dL		
Lac 2.0 mmol/L	Lac 10.0 mmol/L		

(EN)

Product Description

The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions with individual individual bags housed in a 4-cartridge, 3 Calibrators (Calibrator A, B, and F), reference solution, and a flexible waste bag. Each bag includes a septum for a septa. The exposed bag fittings are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use

For the quantitative determination of Na, K, Cl, iCa, Glucose, Lactate, pH, PCO₂, PO₂, and Hct using the Stat Profile Prime VET CCS Comp Analyzer.

Methodology

Refer to Prime Vet CCS Comp Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition

Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

For Veterinary Use Only; Not for Human Use. Refer to the Prime Vet CCS Comp Analyzer Instructions For Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals, Troubleshooting information, Methodology and Principles and other procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

Directions For Use

Refer to the Prime Vet CCS Comp Analyzer Instructions For Use for complete directions for the use of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridges must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards

Analytes traced to NIST Standard Reference Materials.

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