# Stat Profile® Prime Calibrator Cartridge CCS/CCS Comp

Kalibratorkassette, Κασέτα Βαθμονόμησης, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett, Kalibrátorkazetta, キャリブレータ カートリッジ, 측정기 카트리지, 校准液盒

LOT 24066046

2025-08-27

CAL A	$\geq 500 \text{ mL}$	CAL B	$\geq 250 \text{ mL}$	CAL F	$\geq 720 \text{ mL}$	SOLN R	$\geq 300 \text{ mL}$
pH	7.356	pH	6.840	$\text{PCO}_2$	51.0 mmHg	KCl	2 mol/L
$\text{PCO}_2$	24.5 mmHg	$\text{PO}_2$	167.0 mmHg	$\text{PO}_2$	98.0 mmHg		
$\text{Na}^+$	133.0 mmol/L	$\text{Na}^+$	68.5 mmol/L				
$\text{K}^+$	4.0 mmol/L	$\text{K}^+$	10.0 mmol/L				
$\text{Cl}^-$	106.0 mmol/L	$\text{Cl}^-$	46.0 mmol/L				
iCa	1.10 mmol/L	iCa	2.20 mmol/L				
Glu	80 mg/dL	Glu	200 mg/dL				
Lac	2.0 mmol/L	Lac	10.0 mmol/L				

(EN)

**Product Description**  
The above table gives the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridges. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: 3 Calibrators (Calibrator A, B, and F), reference solution, and a flexible waste bag. Each bag includes a fitting with a septa. The exposed bag fittings are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

**Intended Use**  
The Stat Profile Prime Calibrator Cartridge CCS is intended for the calibration of pH,  $\text{PCO}_2$ ,  $\text{PO}_2$ ,  $\text{Hct}$ ,  $\text{Na}^+$ ,  $\text{K}^+$ , iCa, Glucose (Glu), and Lactate (Lac) using the Stat Profile Prime CCS Analyser.

**Methodology**  
Refer to Prime Analyzer Instructions For Use Manual for Test Methodologies and Principles.

**Composition**  
Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M2B-T2).

**Warnings and Cautions:**  
Intended for in vitro diagnostic use.  
Refer to the Prime Analyzer Instructions for Use Manual for complete directions for use, including contraindications, warnings, precautions, calibration intervals, test performance information, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

**Storage**  
Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

**Directions For Use**  
Refer to the Prime Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

**Traceability of Standards**  
Analytes traced to NIST Standard Reference Materials.

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EC REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.