REF 57831, 57832, 57833, 57834, 57835, 65142

Rx only IVD 15°C 30°C CE

Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrerkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιγύο μέσου βαθμονόμησης με κρετινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininné, Στατ Πράιμ Πράιμ Μπλουζ Καρτριζ με θροπνοή Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレーターカートリッジ (クリアチニン), Κρεατίνη Χρήση Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 24075025

2025-09-01

CAL A	CAL B	CAL C	CAL D	CAL F	SOLN R
HCO ₃ 11 mmol/L	pH 6.834 @ 37°C	HCO ₃ 22 mmol/L	pH 7.354 @ 37°C	PO ₂ 101.8 mmHg	KCl 2 mol/L
Na ⁺ 135.5 mmol/L	H ⁺ 146.55 nmol/L	Cl ⁻ 69.0 mmol/L	H ⁺ 44.26 nmol/L	PO ₂ 13.54 kPa	
K ⁺ 4.0 mmol/L	Na ⁺ 77.0 mmol/L	Glu 200 mg/dL			!
Cl ⁻ 108.0 mmol/L	K ⁺ 10.0 mmol/L	Glu 11.10 mmol/L			WASTE CONTAINER W
iCa 1.04 mmol/L	iCa 1.92 mmol/L	Lac 10.0 mmol/L			
iCa 4.16 mg/dL	iCa 7.68 mg/dL	Lac 89.0 mg/dL			
iMg 0.50 mmol/L	iMg 1.50 mmol/L	BUN 50 mg/dL			
iMg 1.22 mg/dL	iMg 3.63 mg/dL	BUN 17.9 mmol/L			
Glu 80 mg/dL	tHb 15.4 g/dL	Creat 5.0 mg/dL			
Glu 4.44 mmol/L	tHb 154 g/L	Creat 0.44 mmol/L			
Lac 2.0 mmol/L	tHb 9.6 mmol/L	Creat 442 μmol/L			
Lac 17.8 mg/dL					
BUN 10 mg/dL					
BUN 3.6 mmol/L					
Creat 1.0 mg/dL					
Creat 0.09 mmol/L					
Creat 88.4 μmol/L					

EN

Description
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use
For quantitative determination of Na⁺, K⁺, Cl⁻, Ca²⁺, Mg²⁺, BUN (Urea), Glucose, Lactate, Creatinine, pH, PO₂, SO₄²⁻, Hct, Hb, HbF, O₂Hb, COHb, MetHb, HHb, and tBil¹ in human blood using the Stat Profile Prime Plus Analyzer.

Methodology
Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

Compatibility
The Cartridges are aqueous salt solutions containing analyses at concentrations listed in the table above. Calibrator B also contains dyes. Contains no constituents of human origin, however, good laboratory practice should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Intended for *in vitro* diagnostic use.
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions for use. Special precautions required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waist Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use
Activation of Cartridge
Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

1. Hold the syringe with tip side down, and remove protective cap.
2. Remove one of the enclosed needle assemblies to the syringe. Remove the protective cover from the needle.
3. Match the color and label of the syringe to the appropriate filament and insert needle.
4. Slowly depress syringe plunger until the contents are dispensed.
5. Press PLUNGER BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
6. Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
7. Repeat Steps 1-5 for the next Cal.
8. Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

Traceability of Standards

Analyses traced to NIST Standard Reference Materials wherever possible.

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