Stat Profile Prime Plus® VET Blood Gas, CO-Oximeter, Chemistry Controls Auto-Cartridge with Creatinine

Cartucho automático con creatinina para controles de química, gases en sangre y cooxímetro Stat Profile Prime Plus® VET, Cartouche automatique de contrôles de gaz du sang/CO-oxymètre, chimie Stat Profile Prime Plus® VET avec créatine, Stat Profile Prime Plus® VET Auto-Kassette mit Kreatinin für Blutgas-, CO-Oximeter- und Blutehemiekontrollen, Αυτόματη συσκευή με κρεατινίνη επιπέδου ελέγχου χημείας ερίστων αίματος, CO-Οξυμέτρο Stat Profile Prime Plus® VET, Cartucioa con creatinina per controlli automatici chimici per gas ematici/CO-ossimetria Stat Profile Prime Plus® VET, Cartucho automático de controles de química, de CO-oxímetro e de gás no sangue Stat Profile Prime Plus® VET com creatinina, Stat Profile Prime Plus® VET vérzgás, CO-oximéter, kémiai kontrollok automatikus patron kretininnel, (מחסנית בקרת כימיה אוטומטית עם קרטינין) Stat Profile Prime Plus® VET Chemistry Controls Auto-Cartridge, Stat Profile Prime Plus® VET 血液ガス, CO オキシメーター, 生化学検査用コントロール自動カートリッジ (クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® VET 혈액 가스, CO-산소 농도계, 화학 조절제 자동 카트리지, Stat Profile Prime Plus® VET 血気、一산화炭-血氣儀、化学对照溶液自动试剂盒 (含肌酐)

LOT 24100110

CONTROL 1 2 3 4 5

2025-09-29

Table with columns for parameter, unit, and expected ranges for controls 1-5. Parameters include pH, H+, PCO2, PO2, SO2, Hct, Na+, K+, Cl-, iCa, iMg, Glu, Lac, BUN, Urea, Creatinine, HbF, tHb, O2Hb, COHb, MelHb, tBil, and iBil.

EN

Product Description: Material espes de control de calidad para supervisar el desempeño de pH, PCO2, PO2, SO2, hematocrito (Hct), hemoglobina fetal (HbF), hemoglobina total (Hb), oxihemoglobina (O2Hb), carboxihemoglobina (COHb), metahemoglobina (MetHb) y desoxihemoglobina (DeHb) en niveles 1, 2 y 3 así como Na+, K+, Cl-, Ca, Mg, Glucose, Lactate, BUN (Urea), Creatinine and Preservatives. Each pouch contains a minimum of 100 mL of reagent solution.

Warnings and Cautions: DO NOT FREEZE. Mix the cartridge by gently inverting for several seconds. DO NOT SHAKE CARTRIDGE. Refer to Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete information.

Storage: Store at 2-8°C (37-46°F). DO NOT FREEZE. Directions for use: Ensure controls are room temperature prior to installation. Level 4 and level 5 must be charged with creatinine syringes prior to installation of the Calibrator Cartridge.

Match the color and label of the syringe to the appropriate ferment and insert needle. Slowly depress syringe plunger until the contents are dispensed. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.

Repeat Steps 1-4 for the next control. My Cartridge will gently invert for 1 minute. Cartridge is ready for use. Verify that the Lot Number on the Expected Ranges Table corresponds to the Lot Number on the cartridge. Refer to Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete details.

Limitations: PO2 values vary inversely with temperature (approximately 1%/°C). Therefore, it is critical to follow the temperature guidelines described in "Directions for Use". The Expected Range values are specific for instruments and controls manufactured by Nova Biomedical.

Traceability of Standards: Total Hemoglobin (Hb) and Methemoglobin (MetHb) are traceable by using Cyanmethemoglobin method. Carboxyhemoglobin (COHb) and Oxymethoglobin (O2Hb) are traceable using Spectrophotometry. Analyses are traced to NIST Standard Reference Materials.

Expected Ranges: The expected range for each parameter was determined at Nova Biomedical using replicate determinations on Nova analyzers. The expected range indicates the maximum deviation from the Mean Value that may be expected under differing laboratory conditions for instruments operating within specifications. Refer to Expected Ranges Table.

INCLSL Document M29-12: 2019 to Define and Determine Reference Intervals in the clinical laboratory; approved guideline-second edition, INCLSL C28-A2, Volume 20, Number 13.

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200 Prospect Street Waltham, MA 02454 U.S.A.