REF 57831, 57832, 57833, 57834, 57835, 65142

Rx Only IVD 15°C 30°C CE

Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrierkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιόγιο μέσω βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibreurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininnel, כוּלֵי עֵם קְרֵאטִינִין Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレータカートリッジ (クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 24129061

2025-10-29

CAL A		CAL B		CAL C		CAL D		CAL F		SOLN R	
HCO ₃	11 mmol/L	pH	6.834 @ 37°C	HCO ₃	22 mmol/L	pH	7.354 @ 37°C	PO ₂	103.6 mmHg	KCl	2 mol/L
Na ⁺	135.5 mmol/L	H ⁺	146.55 nmol/L	Cl ⁻	69.0 mmol/L	H ⁺	44.26 nmol/L	PO ₂	13.78 kPa		
K ⁺	4.0 mmol/L	Na ⁺	77.0 mmol/L	Glu	200 mg/dL						
Cl ⁻	108.0 mmol/L	K ⁺	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.4 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	154 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.6 mmol/L	Creat	442 μmol/L						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 μmol/L										

EN

Description
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use
For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO₂, PO₂, SO₂, Hct, Hb, HbF, O₂Hb, COHb, Methb, Hb, and tHb in human blood using the Stat Profile Prime Plus Analyzer.

Methodology
Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition
The calibrators are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dye. Contains no constituents of human origin, however, good laboratory practices should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Intended for *in vitro* diagnostic use.
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.
Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use
Activation of Cartridge
Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator

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