Stat Profile Prime Plus® VET Blood Gas, CO-Oximeter, Chemistry Controls Auto-Cartridge with Creatinine

Cartucho automático con creatinina para controles de química, gases en sangre y cooxímetro Stat Profile Prime Plus® VET, Cartouche automatique de contrôles de gaz du sang/CO-oxymètre, chimie Stat Profile Prime Plus® VET avec créatine, Stat Profile Prime Plus® VET Auto-Kassette mit Kreatinin für Blutgas-, CO-Oximeter- und Blutchemiekontrollen, Αυτόματα φορτισμένο με κρεατινίνη επιπέδου ελέγχου χημείας αερίων αίματος, CO-Οξυμέτρο Stat Profile Prime Plus® VET, Cartuccia con creatinina per controlli automatici chimici per gas ematici/CO-ossimetria Stat Profile Prime Plus® VET, Cartucho automático de controles de química, de CO-oxímetro e de gás no sangue Stat Profile Prime Plus® VET com creatinina, Stat Profile Prime Plus® VET vérsgáz, CO-oximéter, kémiai kontrollok automatikus patron kreatininnel, (מחמת קבוצת בימה אומטית עם אומטיות ער ארטיאני) Stat Profile Prime Plus® VET Chemistry Controls Auto-Cartridge, Stat Profile Prime Plus® VET 血液ガス, CO オキシメーター, 生化学検査用コントロール自動カートリッジ(クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® VET 혈액 가스, CO-산소 농도계, 화학 조절제 자동 카트리지, Stat Profile Prime Plus® VET 気血、一酸化炭素血気、化学对照溶液自動試劑盒 (含肌肝)

LOT 24117039

CONTROL 1 2 3 4 5

2025-10-17

Table with columns for parameters (pH, H+, Pco2, etc.) and rows for expected ranges across 5 control levels. Includes sub-headers for 'Expected Ranges, Rangos esperados, Plages attendues, Erwartungsbereiche, Αναμενόμενος εύρος, Intervalli previsti, Intervalos previstos, Várt tartományok, הטווחים הצפויים, 予測範囲, 예상 범위, 预期范围值'.

EN

Product Description: Aqueous quality control material for monitoring the performance of pH, Pco2, PO2, SO2, hematoct (Hct), fetal hemoglobin (HbF), total hemoglobin (Hb), oxyhemoglobin (O2Hb), carboxyhemoglobin (COHb), methemoglobin (MetHb), and desoxyhemoglobin (DeHb) in Levels 1, 2 and 3 and all well as Na+, K+, Cl-, Ca, Mg, Glucose, Lactate, BUN (urea), and Creatinine in Level 4 and 5. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

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Limitations: PO2 values vary inversely with temperature (approximately 1% / °C). Therefore, it is critical to follow the temperature guidelines described in "Directions for Use." The Expected Range values are specific for instruments and controls manufactured by Nova Biomedical. Once installed, each Stat Profile Prime Plus VET Cartridge may be used for a maximum of 21 days from the initial installation date on the system at which the system will indicate the cartridge is invalid.

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200 Prospect Street Waltham, MA 02454 U.S.A.