Stat Profile Prime Plus® Blood Gas, CO-Oximeter, Chemistry Controls Auto-Cartridge with Creatinine

Cartucho automático con creatinina para controles de química, gases en sangre y cooxímetro Stat Profile Prime Plus®, Cartouche automatique de contrôles de gaz du sang/CO-oxymètre, chimie Stat Profile Prime Plus® avec créatine, Stat Profile Prime Plus® Auto-Kassette mit Kreatinin für Blutgas-, CO-Oximeter- und Blutchemiekontrollen, Αυτόματο φασίγιμο υλικού ελέγχου χημείας αερίων αίματος, CO-Οξυμέτρο Stat Profile Prime Plus®, Cartuccia con creatinina per controlli automatici chimici per gas ematici/CO-ossimetria Stat Profile Prime Plus®, Cartucho automático de controles de química, de CO-oxímeter e de Gas no sangue Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® vérgáz, CO-oxímeter, kémiai kontrollok automatikus patron kreatininnel, קריאטינין, מוסכמת בקרת כימיה אוטומטית עם קריאטינין Stat Profile Prime Plus® Co-Oxímetro וגז בדם Stat Profile Prime Plus® 血液ガス, CO オキシメーター, 生化学検査用コンロ, 自動カートリッジ (クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® 혈액 가스, CO-산소 농도계, 화학 조절제 자동 카트리지, Stat Profile Prime Plus® 血気、一酸化炭素血気仪、化学対照溶液自動試剂盒 (含肌酐)

LOT 24260064

2026-02-28

CONTROL 1 2 3 4 5

Expected Ranges, Rangos esperados, Plages attendues, Erwartungsbereiche, Αναμενόμενο εύρος, Intervalli previsti, Intervalos previstos, Várt tartományok, הצפויים הטווחים, 予測範囲, 예상 범위, 预期范围

Table with 6 columns: Parameter, CONTROL 1, CONTROL 2, CONTROL 3, CONTROL 4, CONTROL 5. Rows include pH, H+, PCO2, PO2, SO2, Hct, Na+, K+, Cl-, iCa, img, Glu, Lac, Urea, Creatinine, HbF\*, Hb, MetHb, HbH, HbI\*, HbII\*.

EN product Description: Numerous quality control material for monitoring the performance of pH, PCO2, PO2, SO2, hematocrit (Hct), fetal hemoglobin (HbF\*), total hemoglobin (Hb), oxygen hemoglobin (O2Hb), carboxyhemoglobin (COHb), methemoglobin (MetHb), and oxyhemoglobin (HbO2) in Levels 1, 2 and 3 as well as Na+, K+, Cl-, Ca, Mg, Glucose, Lactate, BUN (Urea), and creatinine in Level 4 and 5. Each kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation. For use with Stat Profile Prime Plus Analyzers ONLY.

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Installation: O2 values vary inversely with temperature (approximately 1%/°C). Therefore, it is critical to follow the temperature guidelines specified in "Directions for Use." The Expected Range values are specific for instruments and controls manufactured by Nova Biomedical. Once installed, each Stat Profile Prime Plus Cartridge may be used for a maximum of 21 days from the initial installation but on the system at which time the system will indicate the cartridge is invalid. Each cartridge may be inserted and retrieved from an analyzer a maximum of 6 times.

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EN REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
200 Prospect Street Waltham, MA 02454 U.S.A.