REF 48559, 41549

IVD 15°C 30°C CE

Stat Profile® pHox Ultra/Critical Care Xpress CO-Oximeter Calibrator Cartridge with Bilirubin and Deproteinizing Solution

Kalibrator-Patrone mit Bilirubin und Deproteinisierungslösung, Πακέτο βαθμονομητή με χολερυθρίνη και διάλυμα αποπρωτεϊνισμού, Cartucho de calibrator con bilirubina y solución desproteinizante, Cartouche de calibrateur avec bilirubine et solution de déprotéinisation, Cartuccia calibratore con bilirubina e soluzione deproteinizzante, Cartucho de calibrator com solução de desproteinização e bilirubina, ビリルビンおよび除蛋白液付キヤリブレッタ・カートリッジ, Calibrator Cartridge 와 빌리루빈 및 단백질 제거액, 含有胆红素和去蛋白溶液的定标品盒

LOT 24239036 2025-08-26

Table with 4 columns: CALIBRATOR (tHb, Units), DIL (≥180mL), SOLN (F) (≥620mL), and Deproteinizing Solution (≥90mL). Includes multilingual descriptions and a WASTE CONTAINER warning.

Description For use in calibrating the CO-Oximeter on the Stat Profile pHox Ultra/Critical Care Xpress (CCX) Analyzer. The above table lists the concentrations of the constituents for the calibrator, flush, deproteinizing and diluent solutions contained in the Calibrator Cartridge.

Intended Use For the quantitative determination of total hemoglobin, oxyhemoglobin, carboxyhemoglobin, methemoglobin, deoxyhemoglobin and total bilirubin in human blood using the Stat Profile pHox Ultra/Critical Care Xpress CO-Oximeter Analyzer.

Methodology Refer to the Stat Profile pHox Ultra/Critical Care Xpress Analyzer Instructions for Use Manual for Test Methodologies and Principles.

Composition Solution volumes are listed in the table above. Contains no constituents of human origin, however good laboratory practice should be followed during handling of these materials.

Warnings and Cautions Intended for in vitro diagnostic use. Refer to the Stat Profile pHox Ultra/Critical Care Xpress Analyzer Instructions for Use Manual for complete directions. Follow standard practices for handling laboratory reagents.

Storage Store at 15°C - 30°C. Expiration date is printed on the front of the cartridge.

Directions for Use Refer to the Stat Profile pHox Ultra/Critical Care Xpress Analyzer Instructions for Use Manual for complete directions for installation of the cartridge. Cartridge is ready for use.

Limitations For use on Nova Biomedical Analyzers only.

If not stored at room temperature and out of direct sunlight, the solution may deteriorate. Indications of deterioration may be high or low out of range control values or a noticeable change in calibration or both.

This product is warranted to perform as described in this product insert, and Nova Biomedical disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall Nova Biomedical be liable for any consequential damages arising out of the aforesaid expressed warranty.

Traceability of Standards Traceable to precise hemoglobin measurement using the reference cyanmethemoglobin method.

Reference ICSH, Recommendations for Haemoglobinometry in Human Blood. British Journal of Haematology, 13 (suppl) pp 71-75, 1967.

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