Stat Profile Prime Plus® VET Blood Gas, CO-Oximeter, Chemistry Controls Auto-Cartridge

Cartucho automático para controles de química, gases en sangre y cooxímetro Stat Profile Prime Plus® VET, Cartouche automatique de contrôles de gaz du sang/CO-oxymètre, chimie Stat Profile Prime Plus® VET, Stat Profile Prime Plus® VET Auto-Kassette für Blutgas, CO-Oximeter- und Blutochemiekontrollen, Αυτόματη συσκευή ελέγχου χημικών αερίων αίματος VET, CO-Οξυμέτρο Stat Profile Prime Plus®, Cartuccia per controlli automatici chimici per gas ematici/CO-ossimetria Stat Profile Prime Plus® VET, Cartucho automático de controles de química, de CO-oxímetro e de gás no sangue Stat Profile Prime Plus® VET, Stat Profile Prime Plus® VET vérgáz/CO-oximéter, kémiai kontrollok automatikus patron, CO-Oximeter לברקים כימיים לנו מחטת אוטומטית לברקים כימיים לנו Stat Profile Prime Plus® VET, Stat Profile Prime Plus® VET 血液ガス, CO オキシメーター、化学物質コントロール自動カートリッジ, Stat Profile Prime Plus® VET 혈액 가스, CO-산소 농도계, 화학 조절제 자동 카트리지, Stat Profile Prime Plus® VET 血気、一氧化炭-血氧仪、化学对照溶液自动试剂盒

LOT 24346048

CONTROL 1 2 3 4 5

2026-06-02

Table with columns for parameters (pH, H+, PCO2, PCO2, PO2, SO2, Hct, Na+, K+, Cl-, iCa, iCa, iMg, iMg, Glu, Glu, Lac, Lac, HbF, Hb, Hb, Hb, O.Hb, CO.Hb, Met.Hb, HHb, tBil, tBil) and rows for Control 1, 2, 3, 4, 5. Includes expected ranges and min-max values.

EN

Description product: Aqueous quality control material for monitoring the performance of pH, PCO2, PO2, SO2, hematocrit (Hct), fetal hemoglobin (HbF), total hemoglobin (tHb), oxyhemoglobin (O2Hb), carboxyhemoglobin (COHb), methemoglobin (MetHb), and deoxyhemoglobin (HHb) in levels 1, 2 and 3 as well as Na+, K+, Cl-, Ca, Mg, Glucose, and Lactate in Level 4 and 5. For use with Stat Profile Prime Plus VET Analyzers ONLY.

Intended Use: Intended for monitoring the performance of the Stat Profile Prime Plus VET Analyzers.

Methodology: Refer to Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for Methodology and Principles.

Composition: Controls Levels 1, 2 and 3 are buffered bicarbonate solutions containing dye, salts and preservatives. Each level has a known pH and is equilibrated to a known O2, CO2, and N2 value. Controls Level 4 and 5 are buffered solutions containing known concentrations of Na+, K+, Cl-, Ca, Mg, Glucose, Lactate and preservatives. Each pouch contains a minimum of 100 mL. Controls contain no constituents of human origin, however good laboratory practices should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2)

Warnings and Cautions: DO NOT FREEZE. Mix the cartridge by gently inverting for several seconds, DO NOT SHAKE CARTRIDGE. Refer to Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete information. Follow standard practices for handling laboratory reagents.

Storage: Store at 2-8°C (37-46°F). DO NOT FREEZE.

Directions for use: Ensure controls are at room temperature prior to installation. Mix Cartridge well by gently inverting for 1 minute. Verify that the Lot Number on the Expected Ranges Table corresponds to the Lot Number on the cartridge. Refer to Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions.

Limitations: PO2 values vary inversely with temperature (approximately 1%/°C°F). Therefore, it is critical to follow the temperature guidelines described in "Directions for Use." The Expected Range values are specific for instruments and controls manufactured by Nova Biomedical. Once installed, each Stat Profile Prime Plus VET Cartridge may be used for a maximum of 35 days from the initial installation date on the system at which time the system will indicate the cartridge is invalid. Each cartridge may be inserted and removed from the analyzer a maximum of 6 times.

Traceability of Standards: Total Hemoglobin (tHb) and Methemoglobin (MetHb) are traceable by using Cyanmethemoglobin method. Carboxyhemoglobin (COHb) and Oxyhemoglobin (O2Hb) are traceable using Spectrophotometry. Analyses are traced to NIST Standard Reference Materials.

Reference Intervals: Concentrations are formulated at normal and abnormal expected values in patient blood. The expected clinical range of these values in patient blood is referenced in Tietz, NW ed, 1985 Textbook of Clinical Chemistry, W.B. Saunders Co. Users may wish to determine Mean Values and Expected Ranges in their own laboratory.

Expected Ranges: The expected range for each parameter was determined at Nova Biomedical using replicate determinations on Nova Analyzers. The expected range indicates the maximum deviations from the Mean Value that may be expected under differing laboratory conditions for instruments operating within specifications. Refer to Expected Ranges Table.

NCCLS Document M29-T2: How to Define and Determine Reference Intervals in the clinical laboratory; approved guideline-second edition, NCCLS C28-A2, Volume 20, Number 13.

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200 Prospect Street Waltham, MA 02454 U.S.A.