REF 57831, 57832, 57833, 57834, 57835, 65142

Rx Only IVD 15°C - 30°C CE

# Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrirkassette mit Kreatinin, Stat Profile Prime Plus® Φασίγγιο μέσου βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininell, קריאטינין סטאט פרופיל פריימ פלוס Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレーションカートリッジ (クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 舍肌酐的校准试剂盒, Stat Profile Prime Plus® Kreatinin İçeren Calibrator Cartridge

LOT 24365133

2026-04-17

CAL A		CAL B		CAL C		CAL D		CAL F		SOLN R	
HCO <sub>3</sub>	11 mmol/L	pH	6.834 @ 37°C	HCO <sub>3</sub>	22 mmol/L	pH	7.354 @ 37°C	PO <sub>2</sub>	103.1 mmHg	KCl	2 mol/L
Na <sup>+</sup>	135.5 mmol/L	H <sup>+</sup>	146.55 nmol/L	Cl <sup>-</sup>	69.0 mmol/L	H <sup>+</sup>	44.26 nmol/L	PO <sub>2</sub>	13.71 kPa		W
K <sup>+</sup>	4.0 mmol/L	Na <sup>+</sup>	77.0 mmol/L	Glu	200 mg/dL						
Cl <sup>-</sup>	108.0 mmol/L	K <sup>+</sup>	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.4 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	154 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.6 mmol/L	Creat	442 µmol/L						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 µmol/L										

**EN**  
**Description**  
 The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference Solution, and a flexible waste bag. Each bag includes a fliptop with a septum. The exposed bag fliptops are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the Analyser. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.  
**Intended Use**  
 For the quantitative determination of Na<sup>+</sup>, K<sup>+</sup>, Cl<sup>-</sup>, iCa, iMg, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO<sub>2</sub>, PO<sub>2</sub>, SO<sub>2</sub>%, Hct, Hb, HbF, O<sub>2</sub>Hb, COHb, Methb, Hhb, and tHb in human blood using the Stat Profile Prime Plus Analyzer.  
**Methodology**  
 Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.  
**Composition**  
 The calibrators are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dye. Certain constituents of human origin, however, follow laboratory practice should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).  
**Warnings and Cautions**  
 Intended for in vitro diagnostic use.  
 Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.  
 Waste Bag Contains Preservative (eye, skin, and respiratory irritant).  
**Storage**  
 Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.  
**Directions For Use**  
**Activation of Cartridge**  
 Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the fliptops on the carton for Cal A and Cal C.  
 1. Hold the syringe with tip side down and remove protective cap.  
 2. Attach one of the enclosed needle assemblies to the syringe. Remove the protective cover from the needle.  
 3. Match the color and label of the syringe to the appropriate fliptop and insert needle.  
 4. Slowly depress syringe plunger until the contents are dispensed.  
 DO NOT FULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.  
 5. Remove needle/syringe assembly from fliptop and discard in an appropriate sharps container.  
 6. Repeat Steps 1-5 for the next Cal.  
 7. Mix Cartridge well by gently shaking for 1 minute.  
 Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

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EC REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
200 Prospect Street Waltham, MA 02454 U.S.A.