Stat Profile Prime Plus® Blood Gas, CO-Oximeter, Chemistry Controls Auto-Cartridge with Creatinine

Cartucho automático con creatinina para controles de química, gases en sangre y coxímetro Stat Profile Prime Plus®, Cartouche automatique de contrôles de gaz du sang/CO-oxymètre, chimie Stat Profile Prime Plus® avec créatine, Stat Profile Prime Plus® Auto-Kassette mit Kreatinin für Blutgas-, CO-Oximeter- und Blutchemiekontrollen, Αυτόματο φύσιγγιο υλικού ελέγχου χημείας αερίων αίματος, CO-Οξυμέτρο Stat Profile Prime Plus®, Cartuccia con creatinina per controlli automatici chimici per gas ematici/CO-ossimetria Stat Profile Prime Plus®, Cartucho automático de controles de química, de CO-oxímeter e de gás no sangue Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® vérgáz, CO-oxíméter, kémiai kontrollok automatikus patron kreatininnel, קריאטינין, קוֹ-וֹקְסִימֵטֵר עִם קְרֵיאַטִינִין, מכשיר בקורת כימיה אוטומטית Stat Profile Prime Plus® Co-Oximeter בודד, Stat Profile Prime Plus® Kan Gazı, CO Oksimetre, Kreatininli Kimya Kontrolleri Otomatik Kartuşu, Stat Profile Prime Plus® 血液ガス, CO オキシメーター, 生化学検査用コントロール自動カートリッジ (クレアチニン), 크레아티닌 사용 Stat Profile Prime Plus® 혈액 가스, CO-산소 농도계, 화학 조절제 자동 카트리지, Stat Profile Prime Plus® 血气, 一氧化碳-血氧仪, 化学对照溶液自动试剂盒 (含肌酐)

LOT 25015043

2026-07-08

CONTROL 1 2 3 4 5

Expected Ranges, Rangos esperados, Plages attendues, Erwartungsbereiche, Αναμενόμενο εύρος, Intervalli previsti, Intervalos previstos, Várt tartományok, הצפויים הטווחים, Beklenen Aralıklar, 予測範囲, 예상 범위, 预期范围

Table with 6 columns: Parameter, CONTROL 1 (min - x̄ - max), CONTROL 2 (min - x̄ - max), CONTROL 3 (min - x̄ - max), CONTROL 4 (min - x̄ - max), CONTROL 5 (min - x̄ - max). Rows include pH, H+, PCO2, PCO2, PO2, PO2, SO2, Hct, Na+, K+, Cl-, iCa, iCa, iMg, iMg, Glu, Glu, Lac, Lac, BUN, BUN, Urea, Urea, Creatinine, Creatinine, Creatinine, HbF+, Hb, Hb, Hb, O2Hb, COHb, MetHb, HbH, IBiI+, IBiI+, IBiI+.

EN

Product Description: Aqueous quality control material for monitoring the performance of pH, PCO2, PO2, SO2, hematoct (Hct), fetal hemoglobin (HbF+), total hemoglobin (Hb), total bilirubin (TBil), oxyhemoglobin (O2Hb), desoxyhemoglobin (COHb), methemoglobin (MetHb) and cyanmethemoglobin (CyHb) in Levels 1, 2 and 3 as well as Na+, K+, Cl-, Ca, Mg, Glucose, Lactate, BUN (urea), and Creatinine in Level 4 and 5. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to use. For use with Stat Profile Prime Plus Analyzers ONLY.

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Not available in E.E. U.U. or areas under its purview in the clinical laboratory, approved guideline-second edition, NCCLS C28-A2, Volume 20, Number 13

Made in USA www.novabiomedical.com

EC REP Nova Biomedical GmbH Hensering 13A, Geb. G 64546 Mörfelden-Walldorf Germany

200 Prospect Street Waltham, MA 02454 U.S.A.