REF 57831, 57832, 57833, 57834, 57835, 65142

Rx Only IVD 15°C 30°C CE

Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrerkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιγγιο μέσου βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininnel, Stat Profile Prime Plus® キャリブレータカードリッジ (クリアチニン), 크레아티닌 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒, Stat Profile Prime Plus® Kreatinin içeren Calibrator Cartridge

LOT 25030034

2026-07-21

| CAL A | CAL B | CAL C | CAL D | CAL F | SOLN R |
|------------------------------|------------------------------|-----------------------------|-----------------------------|----------------------------|-------------|
| HCO ₃ 11 mmol/L | pH 6.834 @ 37°C | HCO ₃ 22 mmol/L | pH 7.354 @ 37°C | Po ₂ 103.0 mmHg | KCl 2 mol/L |
| Na ⁺ 135.5 mmol/L | H ⁺ 146.55 nmol/L | Cl ⁻ 69.0 mmol/L | H ⁺ 44.26 nmol/L | Po ₂ 13.70 kPa | |
| K ⁺ 4.0 mmol/L | Na ⁺ 77.0 mmol/L | Glu 200 mg/dL | | | |
| Cl ⁻ 108.0 mmol/L | K ⁺ 10.0 mmol/L | Glu 11.10 mmol/L | | | |
| iCa 1.04 mmol/L | iCa 1.92 mmol/L | Lac 10.0 mmol/L | | | |
| iCa 4.16 mg/dL | iCa 7.68 mg/dL | Lac 89.0 mg/dL | | | |
| iMg 0.50 mmol/L | iMg 1.50 mmol/L | BUN 50 mg/dL | | | |
| iMg 1.22 mg/dL | iMg 3.63 mg/dL | BUN 17.9 mmol/L | | | |
| Glu 80 mg/dL | tHb 15.4 g/dL | Creat 5.0 mg/dL | | | |
| Glu 4.44 mmol/L | tHb 154 g/L | Creat 0.44 mmol/L | | | |
| Lac 2.0 mmol/L | tHb 9.6 mmol/L | Creat 442 μmol/L | | | |
| Lac 17.8 mg/dL | | | | | |
| BUN 10 mg/dL | | | | | |
| BUN 3.6 mmol/L | | | | | |
| Creat 1.0 mg/dL | | | | | |
| Creat 0.09 mmol/L | | | | | |
| Creat 88.4 μmol/L | | | | | |

(EN)

Description

The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D and F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are placed during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use

For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, Creatinine, PCO₂, PO₂%, Hct, tHb, HbF, O₂Hb, COHb, MetHb, Rhb, and tBil in human blood using the Stat Profile Prime Plus Analyzer.

Methodology

Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

Comments

The calibrators are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dyes. Contain no constituents of human origin, however, good laboratory practice should be followed during the handling of these materials. (REF. NCLCS DOCUMENT M29-T2).

Warnings

Intended for in vitro diagnostic use.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use

Activation of Cartridge

Cal A and C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. These two syringes are labeled and color coded to correspond to the filaments on Cal A and Cal C.

- Hold the syringe with tip side down and remove protective cap.
- Attach one of the enclosed needles assemblies to the syringe. Remove the protective needle.
- Match the color and label of the syringe to the appropriate filament and insert needle.
- Slowly depress syringe plunger until the contents are dispensed.
- DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
- Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
- Repeat Steps 1-5 for the next Cal.
- With Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use..

*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

Traceability of Standard

Analytes traced to NIST Standard Reference Materials wherever possible.

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