REF 57838, 57839, 57840

Rx Only IVD 20°C 40°C CE

Stat Profile Prime Plus® Blood Gas, CO-Oximeter, Chemistry Controls Auto-Cartridge

Cartucho automático para controles de química, gases en sangre y cooxímetro Stat Profile Prime Plus®, Cartouche automatique de contrôles de gaz du sang/CO-oxymètre, chimie Stat Profile Prime Plus®, Stat Profile Prime Plus® Auto-Kassette für Blutgas, CO-Oximeter und Blutchemikontrollen, Stat Profile Prime Plus® Αυτομάτο φασίγγιο ελέγχου ελέγχου χημείας αερίων αίματος, CO-Οξυμέτρο, Cartuccia per controlli automatici chimici per gas ematici/CO-ossimetria Stat Profile Prime Plus®, Cartucho automático de controles de química, de CO-oxímetro e de gás no sangue Stat Profile Prime Plus®, Stat Profile Prime Plus® vérgáz/CO-oximéter, Stat Profile Prime Plus® Kan Gaz, CO Oximéter, Kimya Kontrolleri Otomatik Kartuşu, kemiai kontrollok automatikus patron, Stat Profile Prime Plus® בדיקת אוטומטית לנוזל דם Co-Oximeter, Stat Profile Prime Plus® 血液ガス、CO オキシメーター、生化学検査用コントロール自動カートリッジ, Stat Profile Prime Plus® 혈액 가스, CO-산소 농도계, 화학 조절제 자동 카트리지, Stat Profile Prime Plus® 血気、一酸化炭素-血気儀、化学对照液自動試劑盒

LOT 25023039

CONTROL 1 2 3 4 5

2026-04-23

EN

Product Description
Accuracy quality control material for monitoring the performance of pH, PCO2, PO2, SO2, hematocrit (Hct), fetal hemoglobin (HbF), total hemoglobin (Hb), total bilirubin (tBil), xanthoglobulin (Xg), carboxyhemoglobin (COHb), methemoglobin (MetHb), and deoxyhemoglobin (HbD) in Levels 1, 2 and 3, as well as Na+, K+, Cl-, Ca, Mg, Glucose, and Lactate in Level 4 and 5. For use with Stat Profile Prime Plus Analyzers ONLY.

Intended Use
Intended for in vitro diagnostic use by healthcare professionals for monitoring the performance of the Stat Profile Prime Plus Analyzers.

Methodology
Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodology and Principles.

Composition
Controls Levels 1, 2 and 3 are buffered bicarbonate solutions containing dye, salts and preservatives. Each level has a known pH and is equilibrated to a known PO2, CO2 and N2 value. Controls Level 4 and 5 are buffered solutions containing known concentrations of Na+, K+, Cl-, Ca, Mg, Glucose, Lactate and preservatives. Each pouch contains a minimum of 100 mL. Controls contain no constituents of human origin, however good laboratory practices should be followed during handling of these materials.

Warnings and Cautions
DO NOT FREEZE. Mix the cartridge for gently inverting for several seconds. DO NOT SHAKE CARTRIDGE. Refer to Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete information.

Storage
Store at 2-8°C (37-46°F). DO NOT FREEZE.
Directions for use
Ensure controls are at room temperature prior to installation. Mix Cartridge well by gently inverting for 1 minute. Verify that the Lot Number on the Expected Ranges Table corresponds to the Lot Number on the cartridge. Refer to Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions.

Limitations
PO2 values vary inversely with temperature (approximately 1%/°C). Therefore, it is critical to follow the temperature guidelines described in "Directions for Use." The Expected Ranges values are specific for instruments and controls manufactured by Nova Biomedical. Once installed, each Stat Profile Prime Plus Cartridge may be used for a maximum of 35 days from the initial installation date on the system at which time the system will indicate the cartridge is invalid. Each cartridge may be inserted and removed from the analyzer a maximum of 6 times.

Traceability of Standards
Total Hemoglobin (Hb) and Methemoglobin (MetHb) are traceable by using Cyanmethemoglobin method. Carboxyhemoglobin (COHb) and Deoxyhemoglobin (HbD) are traceable using Spectrophotometry. Analyses are traced to NIST Reference Materials.

Reference Intervals
Concentrations are formulated at normal and abnormal expected values in patient blood. The expected clinical range of these values in patient blood is referenced in Table 1W of the 1988 Textbook of Clinical Chemistry, W.B. Saunders Co. Users may wish to determine Mean Values and Expected Ranges in their own laboratory.

Expected Ranges
The expected range for each parameter was determined at Nova Biomedical using replicate determinations on Nova analyzers. The expected range indicates the maximum deviation from the Mean Value that may be expected under differing laboratory conditions for instruments operating within specifications. Refer to Expected Ranges Table.

*Not available in the USA or for Point-of-Care/Non-Patient Testing Use.
INCCLS Document M29-12
How to Define and Determine Reference Intervals in the clinical laboratory, approved guideline-second edition, NCCLS C28-A2, Volume 20, Number 13.

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