# Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrirkassette mit Kreatinin, Stat Profile Prime Plus® Φυσίγγιο μέσου βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininnel, izzatituyuva Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレータカートリッジ (クリアチニン), 크레아티닌 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒, Stat Profile Prime Plus® Kreatinin içeren Kalibrör Kartuşu

LOT 25043035

2026-07-30

CAL A	CAL B	CAL C	CAL D	CAL F	SOLN R
HCO <sub>3</sub> 11 mmol/L	pH 6.834 @ 37°C	HCO <sub>3</sub> 22 mmol/L	pH 7.354 @ 37°C	P <sub>O<sub>2</sub></sub> 104.5 mmHg	KCl 2 mol/L
Na <sup>+</sup> 135.5 mmol/L	H <sup>+</sup> 146.55 nmol/L	Cl <sup>-</sup> 69.0 mmol/L	H <sup>+</sup> 44.26 nmol/L	P <sub>O<sub>2</sub></sub> 13.90 kPa	
K <sup>+</sup> 4.0 mmol/L	Na <sup>+</sup> 77.0 mmol/L	Glu 200 mg/dL			WASTE CONTAINER W
Cl <sup>-</sup> 108.0 mmol/L	K <sup>+</sup> 10.0 mmol/L	Glu 11.10 mmol/L			
iCa 1.04 mmol/L	iCa 1.92 mmol/L	Lac 10.0 mmol/L			
iCa 4.16 mg/dL	iCa 7.68 mg/dL	Lac 89.0 mg/dL			
iMg 0.50 mmol/L	iMg 1.50 mmol/L	BUN 50 mg/dL			
iMg 1.22 mg/dL	iMg 3.63 mg/dL	BUN 17.9 mmol/L			
Glu 80 mg/dL	tHb 15.6 g/dL	Creat 5.0 mg/dL			
Glu 4.44 mmol/L	tHb 156 g/L	Creat 0.44 mmol/L			
Lac 2.0 mmol/L	tHb 9.7 mmol/L	Creat 442 µmol/L			
Lac 17.8 mg/dL					
BUN 10 mg/dL					
BUN 3.6 mmol/L					
Creat 1.0 mg/dL					
Creat 0.09 mmol/L					
Creat 88.4 µmol/L					

## (EN)

## Description

The cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, E), Reference Solution, and a flexible waste bag. Each bag includes its own specific barcode. The enclosed filaments are arranged in a line along the rear of the cartridge tube. They are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

## Intended Use

For the quantitative determination of Na<sup>+</sup>, K<sup>+</sup>, Cl<sup>-</sup>, iCa, Mg BUN (Urea), Glucose, Lactate, Creatinine, PCO<sub>2</sub>, PO<sub>2</sub>, SO<sub>4</sub><sup>2-</sup>, Hct, tHb, HbF, O<sub>2</sub>Hb, COHb, MetHb, HHb, and iBil<sup>1</sup> in human blood using the Stat Profile Prime Plus Analyzer.

## Methodology

Refer to the Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

## Composition

The calibrators are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dyes. Contain no constituents of human origin, however, good laboratory practice should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

## Warnings and Cautions

Intended for in vitro diagnostic use.

Refer to the Stat Profile Prime Plus Analyzer Instructions For Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

## Storage

Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

## Directions For Use

## Activation of Cartridge

Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the cartridge. Cal A and Cal C.

## 1. Hold the syringe with tip side down and remove protective cap.

2. Attach one of the enclosed needle assemblies to the syringes. Remove the protective cover from the needle.

3. Slowly depress syringe plunger until the contents are dispensed.

4. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.

5. Remove needle/syringe assembly from filament and discard in an appropriate sharps container.

6. Repeat Steps 1-5 for the next Cal.

7. Mix cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions For Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

## Traceability of Standards

Analytes traced to NIST Standard Reference Materials wherever possible.

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EC | REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.