REF 57825, 57826, 57827, 57828, 57829, 65141

R_X Only IVD 15°C - 30°C CE

Stat Profile Prime Plus® Calibrator Cartridge

Stat Profile Prime Plus® Kalibrierkassette, Φυσίγγιο μέσου βαθμονόμησης με κρεσόνινη Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus®, Cartuccia calibratore Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Stat Profile Prime Plus® kalibrátor patron, כלי מדידה Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレーションカートリッジ, Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 校正试剂盒, Stat Profile Prime Plus® Calibrator Cartridge

LOT 25083111

2026-09-04

CAL A	CAL B	CAL C	CAL D	CAL F	SOLN R
HCO ₃ 11 mmol/L	pH 6.834 @ 37°C	HCO ₃ 22 mmol/L	pH 7.354 @ 37°C	PO ₂ 105.5 mmHg	KCl 2 mol/L
Na ⁺ 135.5 mmol/L	H ⁺ 146.55 nmol/L	Cl ⁻ 69.0 mmol/L	H ⁺ 44.26 nmol/L	PO ₂ 14.03 kPa	 WASTE CONTAINER W
K ⁺ 4.0 mmol/L	Na ⁺ 77.0 mmol/L	Glu 200 mg/dL			
Cl ⁻ 108.0 mmol/L	K ⁺ 10.0 mmol/L	Glu 11.10 mmol/L			
iCa 1.04 mmol/L	iCa 1.92 mmol/L	Lac 10.0 mmol/L			
iCa 4.16 mg/dL	iCa 7.68 mg/dL	Lac 89.0 mg/dL			
iMg 0.50 mmol/L	iMg 1.50 mmol/L	BUN 50 mg/dL			
iMg 1.22 mg/dL	iMg 3.63 mg/dL	BUN 17.9 mmol/L			
Glu 80 mg/dL	tHb 15.6 g/dL				
Glu 4.44 mmol/L	tHb 156 g/L				
Lac 2.0 mmol/L	tHb 9.7 mmol/L				
Lac 17.8 mg/dL					
BUN 10 mg/dL					
BUN 3.6 mmol/L					

EN

Description
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F SOLN), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use
For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, pH, PCO₂, PO₂, SO₂%, Hct, Hb, HbF, O₂Hb, COHb, MetHb, Hb, and tBil in human blood using the Stat Profile Prime Plus Analyzer.

*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

Methodology
Refer to Stat Profile Prime Plus Analyzer Instructions for Use Manual for Methodologies and Principles.

Composition
The calibrators are aqueous salt solutions containing analytes at concentrations listed in the table above. Calibrator B also contains dye. Contains no constituents of human origin, however, good laboratory practice should be followed during the handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Intended for in vitro diagnostic use.
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.
Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents.

Traceability of Standards
Analytes traced to NIST Standard Reference Materials wherever possible.

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