## Linearity Standard Set A LEVEL 1 2 3 4

Linearity Standard-Set A, Ζύγιολο προτύπων γραμμικότητας A, Conjunto de patrón de linealidad A, Ensemble A étalons de linéarité, Set di standard di linearità A, Padrão de Linearidade - Conjunto A, Linjäritetsstandard Uppsättning A, Linearitási szabvány A. készlet, 直線性基準液 Set A, 직선성 표준 세트 A, 线性标准品套装 A

Expected Ranges, Erwartungsbereiche, Αναμενόμενη περιοχή τιμών, Gammas esperadas, Plages prévues, Intervalli attesi, Intervalos esperados, Förväntade områden, Várható tartományok, 予想範囲, 예상 범위, 预期范围

EN

## Product Description

Aqueous solutions containing Na<sup>+</sup>, Ca<sup>2+</sup>, Mg<sup>2+</sup>, Glu, Lac, K<sup>+</sup>, Cl<sup>-</sup>, HCO<sub>3</sub><sup>-</sup>, Li<sup>+</sup> and Urea. For Use on NOVA analyzers ONLY.

## Intended Use

For *in vitro* diagnostic use with NOVA Stat Profile pHox Plus/L/C/M, Stat Profile pHox Ultra, Stat Profile Prime CCS, Stat Profile Prime ES Plus, Stat Profile Prime ES Comp Plus, and Stat Profile CCX Analyzers to verify calibration, analytical linearity, estimate test imprecision, and detect systematic analytical deviations that may arise from calibrator cartridge or analytical instrument variation.

## Methodology

Refer to Nova Analyzer Instructions for Use for Methodology and Principles of the testing procedures.

## Composition

Buffered solutions containing Na<sup>+</sup>, Ca<sup>2+</sup>, Mg<sup>2+</sup>, Glu, Lac, K<sup>+</sup>, Cl<sup>-</sup>, HCO<sub>3</sub><sup>-</sup>, Li<sup>+</sup> and Urea. Each ampule contains 1.8 ml volume. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

## Warnings and Cautions

Intended for *In Vitro* Diagnostic Use. Cloudiness or particulate matter in the solution is an indication of possible deterioration. DO NOT FREEZE.

Follow standard practices for handling laboratory reagents. Discard in accordance with local requirements.

## Storage

Store at 15-30°C.

## Directions for Use

Linearity Standards are recommended for use as frequently as required by local regulatory and hospital requirements. There are four levels (1, 2, 3, 4).

It is recommended that each laboratory establish the upper and lower limits of its reportable range for each sample type and devise a policy for processing specimens that fall outside of this range.

## Limitations

For use on NOVA analyzers ONLY. Performance characteristics not established for use on other manufacturer's analyzers.

## Traceability of Standards

Analytes traced to NIST Standard Reference Materials.

## Expected Ranges

The expected range indicates the maximum deviations from the mean value that may be expected under differing laboratory conditions for analyzers operating within specifications. The expected ranges for the analytes listed in the table above were verified using replicate determinations on Nova analyzers.

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