REF 52427,52861,53365,53105,53469,53470,52862,52863,53364,53466,53467,53468

Stat Profile® Prime Calibrator Cartridge CCS/CCS Comp

Kalibratorkassette, Κασέτα Βαθμονομητή, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett, Kalibrátorkazetta, キャリブレータ カートリッジ, 측정기 카트리지, 校准液盒, Stat Profile® Prime Calibrator Cartridge CCS/CCS Comp

LOT 26161041

2027-02-19

CAL A ≥ 500 mL			CAL B ≥ 250 mL			CAL F ≥ 720 mL			SOLN R ≥ 300 mL	
pH	7.356		pH	6.840		PCO ₂	51.7	mmHg	KCl	2 mol/L
PCO ₂	25.0	mmHg	PO ₂	167.0	mmHg	PO ₂	98.5	mmHg	WASTE CONTAINER W	
Na ⁺	133.0	mmol/L	Na ⁺	68.5	mmol/L					
K ⁺	4.0	mmol/L	K ⁺	10.0	mmol/L					
Cl ⁻	106.0	mmol/L	Cl ⁻	46.0	mmol/L					
iCa	1.10	mmol/L	iCa	2.20	mmol/L					
Glu	80	mg/dL	Glu	200	mg/dL					
Lac	2.0	mmol/L	Lac	10.0	mmol/L					

EN

Product Description

The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: 3 Calibrators (Calibrator A, B, and F), reference solution, and a flexible waste bag. Each bag includes a fitment with a septa. The exposed bag fitments are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use

The Stat Profile Prime Calibrator Cartridge CCS is intended for the calibration of pH, PCO₂, PO₂, Hct, Na⁺, K⁺, Cl⁻, iCa, Glucose (Glu), and Lactate (Lac) using the Stat Profile Prime CCS Analyzer.

Methodology

Refer to Prime Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition

Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

Intended for in vitro diagnostic use.

Refer to the Prime Analyzer Instructions for Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals, Troubleshooting information, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge.

Directions For Use Refer to the Prime Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards

Analytes traced to NIST Standard Reference Materials.

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