Stat Profile® Prime Calibrator Cartridge ES

Kalibratorkassette, Κασέτα Βαθμονομήτρη, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkassett, Kalibrátorkazetta, キャリブレータ カートリッジ, 측정기 카트리지, 校准液盒

CAL A	$\geq 500 \text{ mL}$	CAL B	$\geq 250 \text{ mL}$	SOLN R	$\geq 300 \text{ mL}$
Na ⁺	135.0 mmol/L	Na ⁺	77.0 mmol/L	KCl	2 mol/L
K ⁺	4.05 mmol/L	K ⁺	10.00 mmol/L		!
Cl ⁻	115.0 mmol/L	Cl ⁻	45.0 mmol/L		WASTE CONTAINER W
iCa	1.00 mmol/L	iCa	2.00 mmol/L		
iMg	0.47 mmol/L	iMg	1.50 mmol/L		

(EN)

Product Description

The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags: Na⁺, K⁺, Cl⁻, iCa, iMg and 2 Calibrators (Calibrator A and B), reference solution, and a waste container bag. Each bag includes a filament with a septa. The exposed bag filaments are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use

The Stat Profile Prime Calibrator Cartridge ES is intended for the calibration of Hct, Na⁺, K⁺, Cl⁻, iCa, and Mg using the Stat Profile Prime ES Analyzer.

Methodology

Refer to Prime Analyzer Instructions For Use Manual for Test Methodologies and Principles.

Composition

Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions:

Intended for *in vitro* diagnostic use.

Refer to the Prime Analyzer Instructions for Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals, Troubleshooting, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guidelines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge. Directions For Use Refer to the Prime Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards

Analytes traced to NIST Standard Reference Materials.

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EC | REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.