REF 57825, 57826, 57827, 57828, 57829

Rx Only IVD 15°C - 30°C CE

Stat Profile Prime Plus® Calibrator Cartridge

Stat Profile Prime Plus® Καλιβριερkασετete, Φυσιγγιο μεσου βαθμονομησης με κρεστινινη Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Cartouche de calibreteurs Stat Profile Prime Plus®, Cartuccia calibratore Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus®, Stat Profile Prime Plus® kalibrator patron, כלי מוסטית Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレータカートリッジ, Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 校准试剂盒

LOT 23089034

2024-08-21

CAL A ≥ 495 mL		CAL B ≥ 250 mL		CAL C ≥ 245 mL		CAL D ≥ 250 mL		CAL F ≥ 850 mL		SOLN R ≥ 500 mL	
HCO ₃	11 mmol/L	pH	6.834 @ 37°C	HCO ₃	22 mmol/L	pH	7.354 @ 37°C	PO ₂	106.1 mmHg	KCl	2 mol/L
Na ⁺	135.5 mmol/L	H ⁺	146.55 nmol/L	Cl ⁻	69.0 mmol/L	H ⁺	44.26 nmol/L	PO ₂	14.11 kPa	WASTE CONTAINER W	
K ⁺	4.0 mmol/L	Na ⁺	77.0 mmol/L	Glu	200 mg/dL						
Cl ⁻	108.0 mmol/L	K ⁺	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.2 g/dL								
Glu	4.44 mmol/L	tHb	152 g/L								
Lac	2.0 mmol/L	tHb	9.4 mmol/L								
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										

EN
Description
 The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F SOLN), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer.
Intended Use
 For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, pH, PCO₂, PO₂, SO₂%, Hct, tHb, HbF*, O₂Hb, COHb, MetHb, HHb, and tBil* in human blood using the Stat Profile Prime Plus Analyzer.
 *Not available in the USA or for Point-of-Care/Near-Patient Testing use.
Methodology
 Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.
Composition
 Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).
Warnings and Cautions
 Intended for *in vitro* diagnostic use. Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).
Storage
 Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.
Directions For Use
 Refer to the Stat Profile Prime Plus Analyzer Instructions for Use for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents.
Traceability of Standards
 Analyses traced to NIST Standard Reference Materials wherever possible.

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EC REP Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.