Stat Profile Prime Plus® VET Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® VET Kalibrierkassette mit Kreatinin, Φυσιγύο μέσου βαθμονόμησης VET με κρεατινίνη Stat Profile Prime Plus®, Cartucho de calibrador con creatinina Stat Profile Prime Plus® VET, Cartucho de calibrador Stat Profile Prime Plus® VET con creatinina, Stat Profile Prime Plus® VET kalibrátor patron kreatinínnel, Stat Profile Prime Plus® VET מבחן כREATינין כויל עם Stat Profile Prime Plus@VET キャリブレータ(クリアチン), 크레이티닌 사용 Stat Profile Prime Plus® VET 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT

23102033 ✓

2024-09-16 ✓

CAL A	≥ 495 mL	CAL B	≥ 250 mL	CAL C	≥ 245 mL	CAL D	≥ 250 mL	CAL F	≥ 850 mL	SOLN R	≥ 500 mL
HCO ₃	11 mmol/L	pH	6.834 @ 37°C	HCO ₃	22 mmol/L	pH	7.354 @ 37°C	PO ₂	104.2 mmHg	KCl	2 mol/L
Na ⁺	135.5 mmol/L	H ⁺	146.55 nmol/L	Cl ⁻	69.0 mmol/L	H ⁺	44.26 nmol/L	PO ₂	13.86 kPa	WASTE CONTAINER	W
K ⁺	4.0 mmol/L	Na ⁺	77.0 mmol/L	Glu	200 mg/dL						
Cl ⁻	108.0 mmol/L	K ⁺	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.2 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	152 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.4 mmol/L	Creat	442 µmol/L						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 µmol/L										

(EN)

Description

The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Cal A, B, C, D and F), Reference Solvent, a Syringe and a Syringe Assembly bag. It includes a needle with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use

For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, Creatinine, pH, PO₂, PO₃, SO₄, Hct, tHb, O₂ Mb, COHb, MetHb, HbB, Hb and tBil in animal whole blood using the Stat Profile Prime Plus VET Analyzer.

Methodology

Refer to Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition

Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions

Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions. Follow standard procedures for handling laboratory reagents. Discard Cartridges according to local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use

Activation of Cartridge

Calibrator Cartridge must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

- Hold the syringe with lip side down and remove protective cap.
- Attach the sterilized needle assembly to the syringe. Remove the protective cover from the needle.
- Match the color and label of the syringes to the appropriate filament and insert needle.
- Slowly depress syringe plunger until the contents are dispensed.
- DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
- Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
- Repeat Steps 1-5 for the next Cal.
- Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards

Analytes traced to NIST Standard Reference Materials wherever possible.

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