REF 58395,58396

15°C - 30°C CE

Stat Profile Prime Plus® VET Calibrator Cartridge

Stat Profile Prime Plus® VET Kalibrierkassette, Φυσιγγίο βαθμονόμησης VET Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® VET, Cartouche de calibreurs Stat Profile Prime Plus® VET, Cartuccia calibratore VET Stat Profile Prime Plus®, Cartucho de calibrador VET Stat Profile Prime Plus®, Stat Profile Prime Plus® VET Kalibrátor patron, Stat Profile Prime Plus® VET כלייל בוסטיל, Stat Profile Prime Plus® VET キャリブレーションカートリッジ, Stat Profile Prime Plus® VET 보정기 카트리지, Stat Profile Prime Plus® VET 校准试剂盒

LOT 23102034

2024-09-16

CAL A		≥ 495 mL	CAL B		≥ 250 mL	CAL C		≥ 245 mL	CAL D		≥ 250 mL	CAL F		≥ 850 mL	SOLN R		≥ 500 mL
HCO ₃	11	mmol/L	pH	6.834	@ 37°C	HCO ₃	22	mmol/L	pH	7.354	@ 37°C	PO ₂	104.2	mmHg	KCl	2	mol/L
Na ⁺	135.5	mmol/L	H ⁺	146.55	nmol/L	Cl ⁻	69.0	mmol/L	H ⁺	44.26	nmol/L	PO ₂	13.86	kPa	! WASTE CONTAINER W		
K ⁺	4.0	mmol/L	Na ⁺	77.0	mmol/L	Glu	200	mg/dL									
Cl ⁻	108.0	mmol/L	K ⁺	10.0	mmol/L	Glu	11.10	mmol/L									
iCa	1.04	mmol/L	iCa	1.92	mmol/L	Lac	10.0	mmol/L									
iCa	4.16	mg/dL	iCa	7.68	mg/dL	Lac	89.0	mg/dL									
iMg	0.50	mmol/L	iMg	1.50	mmol/L	BUN	50	mg/dL									
iMg	1.22	mg/dL	iMg	3.63	mg/dL	BUN	17.9	mmol/L									
Glu	80	mg/dL	tHb	15.2	g/dL												
Glu	4.44	mmol/L	tHb	152	g/L												
Lac	2.0	mmol/L	tHb	9.4	mmol/L												
Lac	17.8	mg/dL															
BUN	10	mg/dL															
BUN	3.6	mmol/L															

EN

Description
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use
For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, pH, PCO₂, PO₂, SO₂ %, Hct, tHb, COHb, MetHb, Hb, and tBil in animal whole blood using the Stat Profile Prime Plus VET Analyzer.

Methodology
Refer to Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition
Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions
Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.
Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use
Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents.

Traceability of Standards
Analyses traced to NIST Standard Reference Materials wherever possible.

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