REF 62225, 62226, 62227, 62228, 62229, 62230

Rx Only IVD 15°C - 30°C

Stat Profile® Prime Calibrator Cartridge ES Plus

Kalibratorkassette, Κασέτα Βαθμονομητή, Cartucho del calibrador, Cartouche étalon, Cartuccia calibratore, Calibrator Cartridge, Kalibratorkasset, Kalibrátorkazetta, キャリブレータ カートリッジ, 측정키 카트리지, 校准液盒

CAL A ≥ 500 mL	CAL B ≥ 250 mL	SOLN R ≥ 300 mL
pH 7.352	pH 6.840	KCl 2 mol/L
Na ⁺ 135.0 mmol/L	Na ⁺ 77.0 mmol/L	WASTE CONTAINER
K ⁺ 4.05 mmol/L	K ⁺ 10.00 mmol/L	
Cl ⁻ 115.0 mmol/L	Cl ⁻ 45.0 mmol/L	
iCa 1.00 mmol/L	iCa 2.00 mmol/L	
iMg 0.47 mmol/L	iMg 1.50 mmol/L	

EN
Product Description
 The above table lists the concentrations of constituents for the calibrators and solutions contained in the Calibrator Cartridge. Contains the following aqueous solutions within individual flexible bags housed in a cardboard box: 2 Calibrators (Calibrator A and B), reference solution, and a flexible waste bag. Each bag includes a firm with a septa. The exposed bag firmets are arranged in a line along the rear of the cartridge box. These septa are pierced during the insertion of the cartridge into the analyzer.

Intended Use
 The Stat Profile Prime Calibrator Cartridge ES Plus is intended for the calibration of pH, Na⁺, K⁺, Cl⁻, iCa, and iMg using the Stat Profile Prime ES Plus and Stat Profile Prime ES Comp Plus Analyzers.

Methodology
 Refer to Prime Analyzer Instructions For Use Manual for Test Methodologies and Principles.
Composition
 Calibrator volumes are listed in the table above. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions:
 Intended for *in vitro* diagnostic use.
 Refer to the Prime Analyzer Instructions for Use Manual for complete directions for use, installation instructions, recommendations for calibration intervals, troubleshooting information, Methodology and Principles of the testing procedures. Follow standard practices required for handling laboratory reagents. Discard Calibrator Cartridges and Waste products in accordance with local guide lines. Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage
 Store at 15-30°C. Expiration date is printed on the front of the Calibrator Cartridge. Directions For Use Refer to the Prime Analyzer Instructions for Use for complete directions for installation of the cartridge, and recommendations for Calibration Intervals. Calibrator Cartridges must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards
 Analytes traced to NIST Standard Reference Materials.

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