REF 58404, 58405

15°C -30°C CE

# Stat Profile Prime Plus® VET Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® VET Kalibrierkassette mit Kreatinin, Φυσιγγιο μέσου βαθμονόμησης VET με κρεατινίνη Stat Profile Prime Plus®, Cartucho de calibrador con creatinina Stat Profile Prime Plus® VET, Cartouche de calibrateurs avec créatinine Stat Profile Prime Plus® VET, Cartuccia calibratore con creatinina Stat Profile Prime Plus® VET, Cartucho de calibrador Stat Profile Prime Plus® VET com creatinina, Stat Profile Prime Plus® VET kalibrátor patron kreatininnel, Stat Profile Prime Plus® VET קריינינין כוונתית ביל עט, Stat Profile Prime Plus® VET キャリブレータ(クレアチニン), 크레이티닌 사용 Stat Profile Prime Plus® VET 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 23047023

2024-07-26

| CAL A            | ≥ 495 mL     | CAL B           | ≥ 250 mL      | CAL C            | ≥ 245 mL     | CAL D          | ≥ 250 mL     | CAL F           | ≥ 850 mL   | SOLN R | ≥ 500 mL                 |
|------------------|--------------|-----------------|---------------|------------------|--------------|----------------|--------------|-----------------|------------|--------|--------------------------|
| HCO <sub>3</sub> | 11 mmol/L    | pH              | 6.834 @ 37°C  | HCO <sub>3</sub> | 22 mmol/L    | pH             | 7.354 @ 37°C | PO <sub>2</sub> | 105.5 mmHg | KCl    | 2 mol/L                  |
| Na <sup>+</sup>  | 135.5 mmol/L | H <sup>+</sup>  | 146.55 nmol/L | Cl <sup>-</sup>  | 69.0 mmol/L  | H <sup>+</sup> | 44.26 nmol/L | PO <sub>2</sub> | 14.03 kPa  |        | <b>WASTE CONTAINER W</b> |
| K <sup>+</sup>   | 4.0 mmol/L   | Na <sup>+</sup> | 77.0 mmol/L   | Glu              | 200 mg/dL    |                |              |                 |            |        |                          |
| Cl <sup>-</sup>  | 108.0 mmol/L | K <sup>+</sup>  | 10.0 mmol/L   | Glu              | 11.10 mmol/L |                |              |                 |            |        |                          |
| iCa              | 1.04 mmol/L  | iCa             | 1.92 mmol/L   | Lac              | 10.0 mmol/L  |                |              |                 |            |        |                          |
| iCa              | 4.16 mg/dL   | iCa             | 7.68 mg/dL    | Lac              | 89.0 mg/dL   |                |              |                 |            |        |                          |
| iMg              | 0.50 mmol/L  | iMg             | 1.50 mmol/L   | BUN              | 50 mg/dL     |                |              |                 |            |        |                          |
| iMg              | 1.22 mg/dL   | iMg             | 3.63 mg/dL    | BUN              | 17.9 mmol/L  |                |              |                 |            |        |                          |
| Glu              | 80 mg/dL     | tHb             | 15.2 g/dL     | Creat            | 5.0 mg/dL    |                |              |                 |            |        |                          |
| Glu              | 4.44 mmol/L  | tHb             | 152 g/L       | Creat            | 0.44 mmol/L  |                |              |                 |            |        |                          |
| Lac              | 2.0 mmol/L   | tHb             | 9.4 mmol/L    | Creat            | 442 μmol/L   |                |              |                 |            |        |                          |
| Lac              | 17.8 mg/dL   |                 |               |                  |              |                |              |                 |            |        |                          |
| BUN              | 10 mg/dL     |                 |               |                  |              |                |              |                 |            |        |                          |
| BUN              | 3.6 mmol/L   |                 |               |                  |              |                |              |                 |            |        |                          |
| Creat            | 1.0 mg/dL    |                 |               |                  |              |                |              |                 |            |        |                          |
| Creat            | 0.09 mmol/L  |                 |               |                  |              |                |              |                 |            |        |                          |
| Creat            | 88.4 μmol/L  |                 |               |                  |              |                |              |                 |            |        |                          |

**EN**

Description  
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrators A, B, C, D and F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Indication Use  
For the quantitative determination of Na<sup>+</sup>, K<sup>+</sup>, Cl<sup>-</sup>, iCa, Mg<sup>2+</sup>, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO<sub>2</sub>, SO<sub>4</sub><sup>2-</sup>, Hct, Hb, O<sub>2</sub>Hb, COHb, MetHb, HHb, and tHb in animal whole blood using the Stat Profile Prime Plus VET Analyzer.

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## Traceability of Standards

Analytes traced to NIST Standard Reference Materials wherever possible.

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**EL**

## Properties

The cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrators A, B, C, D and F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

For the quantitative determination of Na<sup>+</sup>, K<sup>+</sup>, Cl<sup>-</sup>, iCa, Mg<sup>2+</sup>, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO<sub>2</sub>, SO<sub>4</sub><sup>2-</sup>, Hct, Hb, O<sub>2</sub>Hb, COHb, MetHb, HHb, and tHb in animal whole blood using the Stat Profile Prime Plus VET Analyzer.

Methodology  
Refer to Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition  
Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions  
Refer to Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage  
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

## Directions For Use

## Activation of Cartridge

Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

1. Remove the syringes with tip side down and remove protective cap.
2. Attach one of the enclosed needle assemblies to the syringes. Remove the protective cover from the needle.
3. Match the color and label of the syringe to the appropriate filament and insert needle.
4. Gently pull back on the plunger until the contents are dispensed.
5. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
6. Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
7. Repeat Steps 1-5 for the next Cal.
8. Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards  
Analytes traced to NIST Standard Reference Materials wherever possible.

Directions For Use

Activation of Cartridge

Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

1. Remove the syringes with tip side down and remove protective cap.
2. Attach one of the enclosed needle assemblies to the syringes. Remove the protective cover from the needle.
3. Match the color and label of the syringe to the appropriate filament and insert needle.
4. Gently pull back on the plunger until the contents are dispensed.
5. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
6. Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
7. Repeat Steps 1-5 for the next Cal.
8. Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

Traceability of Standards  
Analytes traced to NIST Standard Reference Materials wherever possible.

Directions For Use

Activation of Cartridge

Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

1. Remove the syringes with tip side down and remove protective cap.
2. Attach one of the enclosed needle assemblies to the syringes. Remove the protective cover from the needle.
3. Match the color and label of the syringe to the appropriate filament and insert needle.
4. Gently pull back on the plunger until the contents are dispensed.
5. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
6. Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
7. Repeat Steps 1-5 for the next Cal.
8. Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

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