REF 58395,58396

15°C 30°C CE

Stat Profile Prime Plus® VET Calibrator Cartridge

Stat Profile Prime Plus® VET Kalibrerkassette, Φυσίγγιο βαθμονόμησης VET Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® VET, Cartouche de calibrateurs Stat Profile Prime Plus® VET, Cartuccia calibratore VET Stat Profile Prime Plus®, Cartucho de calibrador VET Stat Profile Prime Plus®, Stat Profile Prime Plus® VET kalibrátor patron, Stat Profile Prime Plus® VET קיליברטור, Stat Profile Prime Plus® VET 보정기 카트리지, Stat Profile Prime Plus® VET 校准试剂盒

LOT 23069027

2024-08-13

CAL A ≥ 495 mL	CAL B ≥ 250 mL	CAL C ≥ 245 mL	CAL D ≥ 250 mL	CAL F ≥ 850 mL	SOLN R ≥ 500 mL
HCO ₃ 11 mmol/L	pH 6.834 @ 37°C	HCO ₃ 22 mmol/L	pH 7.354 @ 37°C	PO ₂ 105.7 mmHg	KCl 2 mol/L
Na ⁺ 135.5 mmol/L	H ⁺ 146.55 nmol/L	Cl ⁻ 69.0 mmol/L	H ⁺ 44.26 nmol/L	PO ₂ 14.06 kPa	
K ⁺ 4.0 mmol/L	Na ⁺ 77.0 mmol/L	Glu 200 mg/dL			
Cl ⁻ 108.0 mmol/L	K ⁺ 10.0 mmol/L	Glu 11.10 mmol/L			
iCa 1.04 mmol/L	iCa 1.92 mmol/L	Lac 10.0 mmol/L			
iCa 4.16 mg/dL	iCa 7.68 mg/dL	Lac 89.0 mg/dL			
iMg 0.50 mmol/L	iMg 1.50 mmol/L	BUN 50 mg/dL			
iMg 1.22 mg/dL	iMg 3.63 mg/dL	BUN 17.9 mmol/L			
Glu 80 mg/dL	tHb 15.2 g/dL				
Glu 4.44 mmol/L	tHb 152 g/L				
Lac 2.0 mmol/L	tHb 9.4 mmol/L				
Lac 17.8 mg/dL					
BUN 10 mg/dL					
BUN 3.6 mmol/L					

(EN)

Description

The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference materials and a suitable waste bag. Each bag includes two glass septa. The exposed bag fittings are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use

For the quantitative determination of Na⁺, K⁺, Cl⁻, iCa, iMg, BUN (Urea), Glucose, Lactate, pH, PCO₂, PO₂, SO₄²⁻, Hct, tHb, O₂Hb, COHb, MetHb, HHb, and tBil in animal whole blood using the Stat Profile Prime Plus VET Analyzer.

Methodology

Refer to Stat Profile Prime Plus VET Analyzer Instructions For Use Manual for Methodologies and Principles.

Composition

Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

Warnings and Cautions

Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

Storage

Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use

Refer to the Stat Profile Prime Plus VET Analyzer Instructions for Use for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents.

Traceability of Standards

Analytes traced to NIST Standard Reference Materials wherever possible.

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