REF 57831, 57832, 57833, 57834, 57835

RX Only IVD 15°C 30°C CE

# Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrerkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιγγίου μέσου βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininnel, סטט למון קרייניניג Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレーターカートリッジ (クレアチニン), 케리아이너 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 23018041

2024-06-15

CAL A	≥ 495 mL	CAL B	≥ 250 mL	CAL C	≥ 245 mL	CAL D	≥ 250 mL	CAL F	≥ 850 mL	SOLN R	≥ 500 mL
HCO <sub>3</sub>	11 mmol/L	pH	6.834 @ 37°C	HCO <sub>3</sub>	22 mmol/L	pH	7.354 @ 37°C	PO <sub>2</sub>	107.8 mmHg	KCl	2 mol/L
Na <sup>+</sup>	135.5 mmol/L	H <sup>+</sup>	146.55 nmol/L	Cl <sup>-</sup>	69.0 mmol/L	H <sup>+</sup>	44.26 nmol/L	PO <sub>2</sub>	14.34 kPa		
K <sup>+</sup>	4.0 mmol/L	Na <sup>+</sup>	77.0 mmol/L	Glu	200 mg/dL						
Cl <sup>-</sup>	108.0 mmol/L	K <sup>+</sup>	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.2 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	152 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.4 mmol/L	Creat	442 µmol/L						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 µmol/L										

(EN)

Description  
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D and F), Reference Solution, and a flexible waste bag. Each bag includes a filament with a septum. The exposed bay filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

Intended Use  
For the quantitative determination of Na<sup>+</sup>, K<sup>+</sup>, Cl<sup>-</sup>, iCa, iMg, BUN (Urea), Glucose, Lacate, Creatinine, pH, PCO<sub>2</sub>, PO<sub>2</sub>, SO<sub>4</sub><sup>2-</sup>, Hct, Hb, HbF<sup>+</sup>, O<sub>2</sub>Hb, COHb, MetHb, HHb, and tBil<sup>+</sup> in human blood using the Stat Profile Prime Plus Analyzer.

Methodology  
Refer to Stat Profile Prime Plus Analyzer Instructions for Use Manual for Methodologies and Principles.

Composition  
Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

Warnings and Cautions  
Intended for *In vitro* diagnostic use.  
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.  
Waste Bag contains Preservative (eye, skin, and respiratory irritant).

Storage  
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

Directions For Use  
Activation of Cartridge  
Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the cartridge. Cal A and Cal C:

- Hold the syringe with tip side down and remove protective cap.
- Attach one of the enclosed needle assemblies to the syringe. Remove the protective cover from the needle.
- Push the plunger until the contents are dispensed.
- Slowly depress the plunger until the contents are dispensed.
- DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
- Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
- Repeat Steps 1-5 for the next Cal.
- With Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

\*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

Traceability of Standards  
Analytes traced to NIST Standard Reference Materials wherever possible.

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Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.