REF 57831, 57832, 57833, 57834, 57835

Rx Only IVD 15°C - 30°C CE

# Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrierkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιγγιο μέσου βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrator patron kreatininnel, Stat Profile Prime Plus® מוחסנת הכילן קרטינין Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレーターカートリッジ (creatine), Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 23047023

2024-07-26

CAL A	≥ 495 mL	CAL B	≥ 250 mL	CAL C	≥ 245 mL	CAL D	≥ 250 mL	CAL F	≥ 850 mL	SOLN R	≥ 500 mL
HCO <sub>3</sub>	11 mmol/L	pH	6.834 @ 37°C	HCO <sub>3</sub>	22 mmol/L	pH	7.354 @ 37°C	PO <sub>2</sub>	105.5 mmHg	KCl	2 mol/L
Na <sup>+</sup>	135.5 mmol/L	H <sup>+</sup>	146.55 nmol/L	Cl <sup>-</sup>	69.0 mmol/L	H <sup>+</sup>	44.26 nmol/L	PO <sub>2</sub>	14.03 kPa	!	WASTE CONTAINER W
K <sup>+</sup>	4.0 mmol/L	Na <sup>+</sup>	77.0 mmol/L	Glu	200 mg/dL						
Cl <sup>-</sup>	108.0 mmol/L	K <sup>+</sup>	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.2 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	152 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.4 mmol/L	Creat	442 µmol/L						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 µmol/L										

**EN**

**Description**  
The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, and F), Reference Solution, and a flexible waste bag. Each bag includes a septa with a needle. The exposed bag filaments are arranged in a line along the rear of the cartridge. The septa are pierced during the insertion of the cartridges into the analyzer. A charge kit consisting of 2 syringes filled with creatinine solution is included for cartridge activation prior to installation.

**Intended Use**  
For the quantitative determination of Na<sup>+</sup>, K<sup>+</sup>, Cl<sup>-</sup>, iCa, iMg, BUN (Urea), Glucose, Lactate, Creatinine, pH, PCO<sub>2</sub>, PO<sub>2</sub> %, Hct, Hb, HbF, O<sub>2</sub>Hb, COHb, MetHb, HHb, and iBil<sup>1</sup> in human blood using the Stat Profile Prime Plus Analyzer.

**Methodology**  
Refer to Stat Profile Prime Plus Analyzer Instructions For Use Manual for Methodologies and Principles.

**Composition**  
Calibrator and solution volumes are listed in the table. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials. (REF. NCCLS DOCUMENT M29-T2).

**Warnings and Cautions**

Intended for *in vitro* diagnostic use.  
Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridges required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

**Storage**  
Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

**Directions For Use****Activating the Cartridge**

Cal A and Cal C must be charged with creatinine syringes prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

1. Remove the syringe with tip side down and remove protective cap.
2. Attach one of the endotracheal needle assemblies to the syringe. Remove the protective cover from the needle.
3. Match the color and label of the syringe to the appropriate filament and insert needle.
4. Press firmly on the plunger until the filament is completely inserted into the syringe.
5. DO NOT PULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
6. Remove needle/syringe assembly from filament and discard in an appropriate sharps container.
7. Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

\*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

**Traceability of Standards**  
Analytes traced to NIST Standard Reference Materials wherever possible.

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