# Stat Profile Prime Plus® Calibrator Cartridge With Creatinine

Stat Profile Prime Plus® Kalibrerkassette mit Kreatinin, Stat Profile Prime Plus® Φυσιγγίο μέσου βαθμονόμησης με κρεατινίνη, Cartucho de calibrador con creatinina Stat Profile Prime Plus®, Cartouche de calibrateurs Stat Profile Prime Plus® avec créatinine, Cartuccia calibratore con creatinina Stat Profile Prime Plus®, Cartucho de calibrador Stat Profile Prime Plus® com creatinina, Stat Profile Prime Plus® kalibrátor patron kreatininnel, יונק ריאתינין עט Stat Profile Prime Plus®, Stat Profile Prime Plus® キャリブレータカートリッジ (ケラチニン), 캐리어타민 사용 Stat Profile Prime Plus® 보정기 카트리지, Stat Profile Prime Plus® 含肌酐的校准试剂盒

LOT 23068042

2024-08-13

CAL A	$\geq 495 \text{ mL}$	CAL B	$\geq 250 \text{ mL}$	CAL C	$\geq 245 \text{ mL}$	CAL D	$\geq 250 \text{ mL}$	CAL F	$\geq 850 \text{ mL}$	SOLN R	$\geq 500 \text{ mL}$
$\text{HCO}_3$	11 mmol/L	pH	6.834 @ 37°C	$\text{HCO}_3$	22 mmol/L	pH	7.354 @ 37°C	$\text{PO}_2$	105.7 mmHg	KCl	2 mol/L
$\text{Na}^+$	135.5 mmol/L	$\text{H}^+$	146.55 nmol/L	$\text{Cl}^-$	69.0 mmol/L	$\text{H}^+$	44.26 nmol/L	$\text{PO}_2$	14.06 kPa		
$\text{K}^+$	4.0 mmol/L	$\text{Na}^+$	77.0 mmol/L	Glu	200 mg/dL						
$\text{Cl}^-$	108.0 mmol/L	$\text{K}^+$	10.0 mmol/L	Glu	11.10 mmol/L						
iCa	1.04 mmol/L	iCa	1.92 mmol/L	Lac	10.0 mmol/L						
iCa	4.16 mg/dL	iCa	7.68 mg/dL	Lac	89.0 mg/dL						
iMg	0.50 mmol/L	iMg	1.50 mmol/L	BUN	50 mg/dL						
iMg	1.22 mg/dL	iMg	3.63 mg/dL	BUN	17.9 mmol/L						
Glu	80 mg/dL	tHb	15.2 g/dL	Creat	5.0 mg/dL						
Glu	4.44 mmol/L	tHb	152 g/L	Creat	0.44 mmol/L						
Lac	2.0 mmol/L	tHb	9.4 mmol/L	Creat	442 $\mu\text{mol/L}$						
Lac	17.8 mg/dL										
BUN	10 mg/dL										
BUN	3.6 mmol/L										
Creat	1.0 mg/dL										
Creat	0.09 mmol/L										
Creat	88.4 $\mu\text{mol/L}$										

**EN****Description**

The Cartridge contains the following aqueous solutions within individual bags: 5 Calibrators (Calibrator A, B, C, D, and F), Reference Solution, and a flexible waste bag. Each bag includes a filter with a septum. The exposed bag filaments are arranged in a line along the rear of the cartridge box. The septa are pierced during the insertion of the cartridge into the analyzer. A charge kit consisting of 5 syringes filled with creatinine solution is included for cartridge activation prior to installation.

**Indended Use**

For the quantitative determination of  $\text{Na}^+$ ,  $\text{K}^+$ ,  $\text{Cl}^-$ ,  $\text{Ca}^{2+}$ ,  $\text{Mg}^{2+}$ ,  $\text{BUN}$  (Urea), Glucose, Lacatate, Creatinine, pH,  $\text{PCO}_2$ ,  $\text{PO}_2$ ,  $\text{SO}_4^{2-}$ ,  $\text{Hct}$ ,  $\text{Hb}$ ,  $\text{HbF}$ ,  $\text{O}_2\text{Hb}$ ,  $\text{COHb}$ ,  $\text{MetHb}$ ,  $\text{HHb}$ , and  $\text{IbI}^*$  in human blood using the Stat Profile Prime Plus Analyzer.

**Methodology**

Refer to Stat Profile Prime Plus Analyzer Instructions for Use Manual for Methodologies and Principles.

**Composition**

Calibrator solution volumes are listed in the table. Contains no constituents of human origin; however good laboratory practice should be followed during handling of these materials. (REF: NCCLS DOCUMENT M29-T2).

**Warnings and Cautions****Intended for In-vitro Diagnostic use**

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents. Discard Cartridge in accordance with local guidelines.

Waste Bag Contains Preservative (eye, skin, and respiratory irritant).

**Storage**

Store at 15 - 30°C. Expiration date is printed on the Calibrator Cartridge.

**Directions For Use****Activation of Cartridge**

Cal A and Cal C must be charged with creatinine syringe prior to installation of the Calibrator Cartridge. The two syringes are labeled and color coded to correspond to the filaments on the carton for Cal A and Cal C.

- Hold the syringe with lip side down and remove protective cap.
- Attach the pre-sterile needle assemblies to the syringe. Remove the protective cap from the needle.
- Match the color and label of the syringe to the appropriate filament and insert needle.
- Slowly depress syringe plunger until the contents are dispensed.
- DO NOT FULL BACK ON THE PLUNGER TO FLUSH CONTENTS OF SYRINGE.
- Remove needles/syringe assembly from filament and discard in an appropriate sharps container.
- Repeat Steps 1-5 for the next Cal.
- Mix Cartridge well by gently shaking for 1 minute.

Refer to the Stat Profile Prime Plus Analyzer Instructions for Use Manual for complete directions. Calibrator Cartridge must be inverted gently several times to thoroughly mix contents. Cartridge is ready for use.

\*Not available in the USA or for Point-of-Care/Near-Patient Testing use.

**Traceability of Standards**

Analytes traced to NIST Standard Reference Materials wherever possible.

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Nova Biomedical GmbH Hessenring 13A, Geb. G 64546 Mörfelden-Walldorf Germany
Waltham, MA 02454 U.S.A.