Stat Profile® Prime Auto QC Cartridge CCS

Kontroll-Auto-Kassetten, Αυτόματη Κασέτα Διαλυμάτων Ελέγχου, Cartucho automático para controles, Cartouche automatique controlli, Cartucho automático de control, Kontroller, autokassett, Automatikus minősé-ellenőrző kazetta, 自動QCカートリッジ, 자동QC카트리지, 自动质控盒

CONTROL 1 2 3

LOT 23047067 2024-08-02

Table with 4 columns: Parameter, Unit, CONTROL 1 (min - x̄ - max), CONTROL 2 (min - x̄ - max), CONTROL 3 (min - x̄ - max). Rows include pH, H+, pCO2, pCO2, pO2, pO2, Hct, Na+, K+, Cl-, iCa, iCa, Glu, Glu, Lac, Lac.

EN Product Description: Consists of 3 flexible bags within a cardboard carton. Each bag contains an aqueous quality control material for monitoring the measurement of pH, PCO2, PO2, hematocrit (Hct), Na, K, Cl, Ca, Glucose (Glu) and Lactate (Lac) for use with Nova Biomedical analyzers ONLY.

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Methodology: Refer to Prime Analyzer Instructions For Use Manual. Composition: A buffered bicarbonate solution, each control with a known pH and known levels of Na, K, Cl, iCa, Glu and Lac.

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Warnings and Cautions: Must be stored at 24-28°C for at least 24 hours before use. DO NOT FREEZE. Intended for in vitro diagnostic use.

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Directions for use: Cartridge must be stored at approximately 24-28°C for at least 24 hours prior to opening. Refer to Analyzer Instructions for Use for complete instructions.

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Traceability of Standards: Reference Intervals: Concentrations are formulated to represent three pH levels (Acidosis, Normal pH, and Alkalosis).

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Expected Ranges: The EXPECTED RANGE for each analyte was determined at Nova Biomedical by using multiple runs of each level of control at 37°C on multiple instruments.

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How to Define and Determine Reference Intervals in the clinical laboratory; approved guideline-second edition, NCLLS C28-A2, Volume 20, Number 13

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