StatProfile Prime Ampule Control ES

CONTROL 1 2

LOT 22243083

2024-08-09

Expected Ranges, Erwartungsbereiche, Αναμενόμενα εύρη, Intervalos esperados, Plages attendues, Intervalli previsti, Várható tartományok, 預測範圍, 예상 범위, Gamas previstas, 预期范围			
LOT		CONTROL 1	CONTROL 2
CONTROL 1	22221037	2024-08-09	min - \bar{x} - max
CONTROL 2	22221038	2024-08-09	min - \bar{x} - max
pH*		7.39 - 7.44 - 7.49	7.51 - 7.56 - 7.61
H+*	nmol/L	40.7 - 36.3 - 32.4	30.9 - 27.5 - 24.5
Hct	%	48 - 51 - 54	64 - 67 - 70
Na+	mmol/L	141.4 - 146.4 - 151.4	114.6 - 118.1 - 121.6
K+	mmol/L	3.44 - 3.64 - 3.84	6.01 - 6.31 - 6.61
Cl-	mmol/L	101.7 - 105.7 - 109.7	80.0 - 84.0 - 88.0
iCa	mg/dL	4.05 - 4.69 - 5.33	6.77 - 7.58 - 8.38
iCa	mmol/L	1.01 - 1.17 - 1.33	1.69 - 1.89 - 2.09
iMg	mg/dL	0.75 - 1.07 - 1.39	2.48 - 2.92 - 3.35
iMg	mmol/L	0.31 - 0.44 - 0.57	1.02 - 1.20 - 1.38

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EN

Product Description

Contains assayed quality control material formulated at three levels for monitoring the measurement of pH, Hct, Na+, K+, Cl-, Li+, Ca++, and Mg++.

Intended Use

For *in vitro* diagnostic use to monitor the performance of the Stat Profile Prime ES, Stat Profile Prime ES Plus, and Stat Profile Prime ES Comp Plus Analyzers.**

Methodology

Refer to StatProfile Prime Analyzer Instruction for Use Manual for Methodology and Principles.

Composition

Nova Control is a pale straw colored liquid assayed quality control material. Contains organic and inorganic constituents in a bovine albumin base. A preservative has been added to the product; however it does not contain stabilizers, viscosity adjusters, drug analogs or other materials which might affect the performance of sensors. Controls contain a buffered solution with known concentrations of pH, Hct, Na+, K+, Cl-, Li+, Ca++, and Mg++.

Each ampoule contains a minimum volume of 1.2 mL. Contains no constituents of human origin, however, good laboratory practice should be followed during handling of these materials.¹

Warning and Precautions

For *in vitro* diagnostic use. Store at 2-8°C. Do not Freeze. Refer to the Nova Instructions for Use Manual for complete directions. Follow standard practices required for handling laboratory reagents.

Storage

Store at 2-8°C. Do not freeze. Each ampoule has a lot number and expiration date printed on the label.

Directions for Use

Re-suspend the ampule contents. Nova Controls contain proteins, which may settle with storage. Gently rotate each ampule between fingers or hands to mix contents prior to use. Avoid shaking, excess foaming may occur. Snap open the ampule (protecting fingers with gauze or gloves), aspirate solution with sample probe or transfer solution into sample cups. Verify that the lot number appearing on the expected ranges table is the same as the lot number printed on the material.

Limitations

Assigned day values will be adversely affected by improper handling or storage, or if the integrity of the product packaging has been compromised prior to use. Cloudiness or particulate matter in the solution is an indication of possible deterioration.

For use on Nova Biomedical Analyzers Only. Performance characteristics have not been established for use on other manufacturer's analyzers.

This product is warranted to perform as described in this product insert, and Nova Biomedical disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall Nova Biomedical be liable for any consequential damages arising out of the aforesaid expressed warranty.

Traceability of Standards

Analytes are traced to NIST Standard reference materials.

NCCLS Document M29-T2

**The Stat Profile Prime ES Plus and Stat Profile Prime ES Comp Plus Analyzers are not currently CE marked.

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